CE 0123  
FC

UK  
CA

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# ENGLISH

|                                                   |           |
|---------------------------------------------------|-----------|
| <b>1. KEY FOR OVERVIEW</b>                        | <b>31</b> |
| <b>2. PREFACE</b>                                 | <b>31</b> |
| <b>3. PRODUCT DESCRIPTION</b>                     | <b>31</b> |
| 3.1. SCOPE OF DELIVERY                            | 31        |
| 3.2. DEVICE TYPE NOVAFON max+                     | 31        |
| 3.3. ACCESSORIES FOR NOVAFON max+                 | 32        |
| 3.3.1. DISC ATTACHMENT                            | 32        |
| 3.3.2. BALL ATTACHMENT                            | 32        |
| 3.3.3. CHARGING STAND                             | 32        |
| 3.3.4. OPTIONAL ACCESSORIES                       | 33        |
| 3.3.5. COMBINATION WITH OTHER PRODUCTS            | 33        |
| <b>4. OPERATION</b>                               | <b>33</b> |
| 4.1. DISPLAY                                      | 33        |
| 4.2. INITIAL START-UP AND FREQUENCY               | 34        |
| 4.3. SETTING THE INTENSITY/STRENGTH               | 35        |
| 4.4. CHARGING                                     | 35        |
| 4.5. THE APP/BLUETOOTH                            | 36        |
| 4.6. ATTACHMENTS                                  | 37        |
| <b>5. APPLIKATION &amp; USE &amp; USER GROUPS</b> | <b>37</b> |
| 5.1. INTENDED USE                                 | 38        |
| 5.2. GENERAL INFORMATIONS FOR USE                 | 38        |
| 5.3. SELF-APPLICATION AT HOME                     | 39        |
| 5.4. PROF. APPLICATION IN NEURO-REHABILITATION    | 39        |
| 5.4.1. SPASTICITY                                 | 40        |
| 5.5. PROF. USE IN VOICE AND SWALLOW THERAPY       | 40        |
| <b>6. CONTRAINDICATIONS</b>                       | <b>40</b> |
| <b>7. TECHNICAL SPECIFICATIONS</b>                | <b>41</b> |
| 7.1. SYMBOLS USED                                 | 41        |
| 7.2. TECHNICAL SPECIFICATIONS                     | 42        |
| <b>8. IMPORTANT SAFETY INSTRUCTIONS</b>           | <b>44</b> |
| 8.1. WARNINGS                                     | 44        |
| 8.2. OTHER PRECAUTIONS                            | 48        |
| <b>9. MAINTENANCE AND REPAIR</b>                  | <b>49</b> |
| 9.1. CLEANING AND DISINFECTION                    | 49        |
| 9.2. TROUBLESHOOTING                              | 49        |
| 9.2.1 FACTORY SETTINGS                            | 50        |
| 9.3. SAFETY CHECK, MAINTENANCE AND REPAIR         | 50        |
| 9.4. FIRMWARE UPDATES                             | 50        |
| 9.5. DISPOSAL                                     | 51        |
| <b>10. WARRANTY</b>                               | <b>51</b> |
| <b>11. CUSTOMER SERVICE</b>                       | <b>52</b> |

## 1. KEY FOR OVERVIEW

On page 1 you will find an overview of your device

- 1 Attachment
- 2 Display
- 3 Up
- 4 ON / STANDBY / OFF  
100 Hz / 75 Hz / 50 Hz
- 5 Down
- 6 Charge connection

## 2. PREFACE

Congratulations on your purchase of a NOVAFON vibration sound wave device. Our therapy devices have delivered excellent performance and reliability for decades. Please read the following instructions carefully to ensure that you can get the most out of your NOVAFON vibration sound wave device and enjoy using it for a long time. The explanation of the footnotes can be found on the last page of this instructions.

## 3. PRODUCT DESCRIPTION

### 3.1. SCOPE OF DELIVERY

Delivery includes a battery powered NOVAFON max+ sound wave device as well as a disc attachment, a ball attachment, a power plug, a charging cable and instructions for use.

At the beginning of the instructions for use you will also find instructions on how to download the NOVAFON App – for optimal use of your NOVAFON device.

### 3.2. DEVICE TYPE NOVAFON max+

NOVAFON vibration sound wave devices are designed for local vibration therapy. Different attachments are used to transmit the mechanical vibrations into the tissue. Thanks to its safe and simple application, local vibration therapy can be used in many ways. Thanks to the three treatment frequencies (50 Hz, 75 Hz und 100 Hz) featuring five intensity settings each (low to high) treatment can be individually adapted to the body part and underlying symptoms in question. The following table describes the characteristics of your new NOVAFON de-

vice NOVAFON max+.The following table describes the characteristics of your new NOVAFON device NOVAFON max+.

|                                                                                                              |                                                                                                                                                                                                                                                                    |
|--------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Multifunction buttons</b>                                                                                 | ON/OFF  and  ,  |
| frequency settings          | 50 Hz, 75 Hz, 100 Hz,<br>Stand-by, Off                                                                                                                                                                                                                             |
| Intensity/strength setting  | low  to high                                                                                     |
| <b>Intensity</b>                                                                                             | ●●●●●                                                                                                                                                                                                                                                              |
| <b>Depth of penetration</b>                                                                                  | 9 cm                                                                                                                                                                                                                                                               |
| <b>Colour</b>                                                                                                | black                                                                                                                                                                                                                                                              |
| <b>Lithium ion battery</b>                                                                                   | Capacity 5000 mAh, Voltage<br>3,6 V, 18 Wh, Type 21700<br>with protection board                                                                                                                                                                                    |

Charging time approx. 3.5h, Battery life up to 3.5h depending on frequency and intensity.

For more technical specifications please see the technical data sheet in the download section of our website.

### 3.3. ACCESSORIES FOR NOVAFON max+

NOVAFON vibration sound wave devices are equipped with two interchangeable attachments as standard. NOVAFON GmbH also offers other attachments that are not included in this set. Please contact your dealer or visit [www.novafon.com](http://www.novafon.com) for more information. NOVAFON devices may only be used with original NOVAFON accessories.

The use of accessories from other manufacturers can impair the function and performance of NOVAFON devices.

#### 3.3.1. DISC ATTACHMENT

The disc attachment is used in the treatment of larger body parts that require broader strokes. It is supplied as standard and also available as a spare part.

#### 3.3.2. BALL ATTACHMENT

The ball attachment can be used for smaller areas and targeted treatment. It is supplied as standard and also available as a spare part.

#### 3.3.3. CHARGING STAND

The charging stand can be purchased together with the device or separately. The charging stand offers convenient storage for your device during and after charging. These instructions for use include all the information you need on how to use and clean your stand, as well as any necessary precautionary measures. The charging stand is delivered with an additional charging cable.

#### 3.3.4. OPTIONAL ACCESSORIES

On our website you will find additional attachments for various treatment goals.

NOTE: Depending on your country, the device may also be equipped with a second power plug with country-specific connections. These are also available as spare parts.

#### 3.3.5. COMBINATION WITH OTHER PRODUCTS

NOVAFON devices are not intended to be used with medical products other than NOVAFON original accessories.

## 4. OPERATION

### 4.1. DISPLAY

The display shows everything you need to know about the status

#### Information on device status

##### View 1

Battery status  
(only displayed at the beginning of treatment and when the battery is low)

Bluetooth status

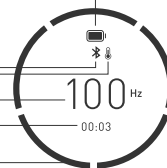
overheating protection active

Frequency

Runtime (current use)

Intensity ring

(see chapter 4.2, 4.3, 4.4 , 4.5)

### View 2

Travel-Lock – Device locked / Switch to unlocked (see chapter 4.1)

### View 3

Welcome screen  
(see chapter 4.1)

TIP: Use the app for  
customisation!  
Information during use

## 4.2. INITIAL START-UP AND FREQUENCY

Fully charge the device prior to switching it on for the first time (chapter 4.4). The device is delivered in locked status and must first be unlocked. Unlock the device by pressing and holding down the ON/OFF button for 3 seconds. You will now see the welcome text (view 3), followed by the app screen (battery status, frequency, runtime; see view 1). The device starts operation at 100 Hz. Press the ON/ OFF-button again to reduce the frequency to 75 Hz, 50 Hz or 0 Hz respectively. Your current frequency setting is shown on the display (see view 1). To switch the device off, press and hold the ON/OFF button again. After about 3 seconds the device will switch off. The duration of the current treatment will be shown on the display screen before the display switches off after a further 3 seconds.

If you are carrying your NOVAFON with you, we recommend locking the device to prevent it from accidentally switching itself on while in your bag/case. To do this, press the ON/OFF button until you see an open lock and a circle closing around the display. As soon as the circle closes completely, the lock will also be displayed as closed. The display will then switch off. The device switches off briefly after 20 minutes of use. Devices are designed for 20 min use, 15 min break. Longer use than 20 min has no medical advantage;

the expected effect (pain relief, improvement in motor skills etc.) is often achieved after 5 to 10 min use.

## 4.3. SETTING THE INTENSITY/STRENGTH

You can regulate the intensity of your device by pressing the Intensity buttons UP or DOWN. Here you can choose from three intensity settings: low (○) to high (○). You can check the intensity setting by looking at the ring around the display (see view 1). The device is set to start on the lowest intensity setting by default (○). This also corresponds to NOVAFON's recommendations for treatment i.e. start at low intensity and increase as required, according to each individual's sensitivity.

TIP: Using the NOVAFON app you can adapt the intensity within the individual levels.

NOTE: The best vibration is achieved when the device is operated without additional contact pressure. If your device automatically switches to a lower setting, or you are unable to increase the intensity, this is probably due to a low battery level. In this case, you should charge your device.

NOTE: In case excessive pressure is applied, the drive will lock; a click can be heard and the vibration stops. The vibration can also stop without resting by applying a lot of pressure.

If you use the device only at the highest intensity, the over-heating protection may be activated to protect the device from damage. The symbol (⚠) appears on the display. When the symbol appears, you can continue to use the device, but no longer at the highest intensity. The device automatically switches to a lower intensity where temperatures are uncritical. Usually, you can continue to work at lower intensities; however, in exceptional cases the device can switch off completely. Please note that the devices are generally designed for short-term operation, which means that after 20 minutes of use, the devices should be switched off for at least 15 minutes.

## 4.4. CHARGING

To charge the device, you should only use the supplied power plug and cable, or the NOVAFON charging stand if included (depending on the model, the charging stand may not be included in delivery). To charge your device, first connect the cable to the device or charging stand and then to the

power plug suitable for your power connection. Only then should you plug the power plug into the socket. Note that only the cable supplied with the charging stand can be used to connect the charging stand. To charge the device without a charging stand, use the charging cable included in the device packaging. Charge the device until the battery is fully charged (see battery icon, view 1). Do not use the NOVAFON while it is charging. The device has a low battery indicator. When the low battery indicator appears (see view 1), you can continue to use your NOVAFON for a few minutes before it switches off automatically. When using the charging stand to charge your device, make sure you are careful when placing the device onto the connector and removing it. To remove, hold the charging stand and gently release the device. The device is charging when the battery indicator appears (see view 1). Avoid twisting the device in the charging station in order to prevent damage.

To avoid malfunctions, the device must be completely dry and free from residues of cleaning agents, disinfectants or oils/creams.

NOTE: When the device is fully charged, disconnect the power plug and device from the electrical supply to avoid damage. If the device is connected to the charging accessory, all pole breaking and complete disconnection from the mains can only take place if the mains plug is removed from the socket.

#### 4.5. THE APP/BLUETOOTH

The Bluetooth mode is deactivated or activated, by pressing the UP-button for 5 seconds. To pair your device is ready with your mobile device and the NOVAFON app. Follow the instructions in the app using the "My NF" tab. Press the intensity and frequency buttons on the device at the same time until "Searching" appears on the display. The color of the Bluetooth icon (view 1) changes if the process was successful. You will now see 2 small triangles to the left and right of the Bluetooth icon. In order to pair the device with another app (e.g. second cell phone, tablet...), the previous connection must be terminated by simultaneously pressing and holding the intensity and frequency buttons. Wait 5 seconds to reconnect. To deactivate the Bluetooth mode, press and hold the intensity button again for approx. 5 seconds.

The app allows you to adjust additional settings on your device:

- Customisation of display colours
- Overview of treatment durations/usage statistics
- Device information display (serial number, firmware version)
- Favourite frequency and intensity settings
- Customised welcome text on the display screen
- Firmware updates, if available, an update is only possible via the app

NOTE: You can also use the device without using the app. However, you will then only be able to access the basic features described in these instructions for use.

#### 4.6. ATTACHMENTS

The attachments are fixed to the device via a plug connection. They can easily be detached by turning them a quarter turn and pulling them gently off the device. The attachments click simply onto the device. The attachments are in the correct position when the brackets on the device and the attachment lie seamlessly on top of each other. The magnets help you to find the final position. Before starting treatment, make sure that the respective attachment is engaged in the intended final position. Due to the wave design the attachments can be individually adjusted by 90° thus allowing individual treatment direction and application to your needs. The direction of the attachment has no influence on the transmission of the vibrations into the tissue.

Please refer to the drawing on page 1 for the correct position of the attachment on the device.

### 5. APPLICATION & USE & USER GROUPS

NOVAFON sound wave devices are used by doctors and therapists to support their treatment in occupational therapy, physiotherapy and speech therapy. Thanks to their ease of use, the devices can also be used in the home environment (home use by lay users). Training in how to use the devices is therefore not required. The information in these instructions for use must be observed. Please note that NOVAFON sound wave devices only alleviate symptoms. They do not treat the underlying cause and are not a substitute for visiting a doctor. Despite their safe,

simple, and non-invasive application, we suggest that you ideally discuss the treatment with your treating doctor or therapist. If symptoms worsen, please seek medical advice. Stop the treatment immediately if you feel dizzy, unwell, or feel skin irritation or increased pain. Please always note the contraindications listed in chapter 6.

Please note that some applications may require the use of special NOVAFON accessories (sold separately). Devices and accessories can be used from head to toe. **DO NOT USE ON THE GENITALIA.**

NOVAFON max+ devices are intended for treatment of people from the age of 13. Use on younger patients may also be possible if monitored by specialist staff. However, **DO NOT** use NOVAFON devices on patients under the age of 2. When using the device on children and adolescents, be sure to work only at low intensities.

### 5.1. INTENDED USE

NOVAFON devices and accessories are designed to relieve pains and aid in relaxation of muscles<sup>1</sup> to alleviate symptoms of:

- Temporary and transient pain of the musculoskeletal system<sup>1</sup>
- Chronical pain of the musculoskeletal system<sup>1</sup>
- Neuro Rehabilitation (sensorimotor disorders)<sup>2,3</sup>
- Voice and swallowing disorders<sup>4</sup>

Professional users: Occupational therapists, physio therapists, speech therapists and the like. Home use by lay users, mostly in self-application, but also relatives and caretakers to mobility impaired patients

**NOTE:** When using the NOVAFON max+ on sensitive parts of the body, such as the neck, face and hands, make sure to only work with low intensities.

### 5.2. GENERAL INFORMATIONS FOR USE

The different frequencies have different effects, so 100 Hz<sup>1</sup> is used in particular to reduce pain, but also to activate muscles. 75 Hz, in particular, increase the muscle excitation potential<sup>6</sup> and thus enable muscle activation and training preparation, especially in everyday sports, 50 Hz, on the other hand, serve to detonate the muscles and thus

help in general relaxation, but also in relaxation of spastic muscles.

Treatment should not exceed a maximum of 20 minutes per session, nor a maximum of 3 sessions per day. Please note that after 20 minutes of use, the NOVAFON should be switched off for 15 minutes in order to avoid overheating. The device automatically switches to stand-by mode after 20 minutes of continuous use. When treating severely painful or sensitive parts of the body (e.g. face, neck, head), you should reduce/adjust the duration of treatment according to each individual's sensitivity.

**NOTE:** Application of liniments: Apply only a thin layer of creams, ointments or cosmetic/medicinal liniments to the skin, or place a fine cloth between the device and the skin. Do not allow liquids or residues to penetrate the NOVAFON as this may cause damage. After using agents of this type, make sure you clean the device thoroughly.

**NOTE:** Discontinue treatment immediately if side effects occur [chapter 6].

### 5.3. SELF-APPLICATION AT HOME

Taking into account the contraindications (chapter 6), move the NOVAFON device in a straight line, selectively or in slowly circling movements on or around the treatment area. The NOVAFON device can also be applied at trigger points. In case of uncertainty, please consult your treating physician or therapist regarding your individual application. The NOVAFON device can be used directly on the skin or through fabric/clothing. During the application, make sure that the vibration sound wave device glides gently and without pressure over the body area. Use the entire surface of the attachment. This is the only way the penetration depth effect of the NOVAFON device can be fully deployed and the vibrations are transmitted optimally. Use only the NOVAFON device's own weight for a comfortable application.

More detailed information on possible treatment sequences can be found in our application book (enclosed with the delivery or online in our download area at [www.novafon.com](http://www.novafon.com))  
**TIP:** With the APP you can save your favorite combination of frequency and intensity permanently and call it up directly for each application.

## 5.4. PROFESSIONAL APPLICATION IN NEURO-REHABILITATION

NOTE: The use of local vibration therapy for neurological diseases should only be carried out by specialists (therapists, physicians, nurses) or after detailed instruction by such a person.

The general application descriptions from the previous chapters also apply to professional use.

### 5.4.1. SPASTICITY

Spasticity is a common consequence of a stroke and other disorders of the central and peripheral nervous system. Local vibration therapy, combined with physiotherapy, provides a simple and drug-free treatment method. Numerous studies have shown a reduction in spasticity as well as improved motor control and function.<sup>2</sup> The effectiveness mechanisms are the tonic vibrational reflex and the stimulation of the sensomotoric system. Taking into account the contraindications (chapter 6), the spastic muscle itself is treated at 50 Hz and the antagonist of the spastic muscle at 100 Hz. The starting point can be on the tendon or directly on the muscle itself. The treatment may be local, i.e. focusing on one point, or in sweeping movements. The application with the NOVAFON device shall not exceed 20 minutes per unit. The treatment can be performed up to 5 times a week. Please keep in mind that using the appliance incorrectly can lead to increased muscle tone. Stop the treatment immediately if any side effects occur (see chapter 6).

## 5.5. PROFESSIONAL USE IN VOICE AND SWALLOW THERAPY

NOTE: The use of local vibration therapy for swallowing and voice disorders must only be carried out by specialists (therapists, doctors, nursing staff) or after detailed instruction by specialists. The general application descriptions from the previous chapters also apply to professional use.

NOTE: NOVAFON max+ is primarily suitable for the treatment of malposition (tension, pain) and general posture in voice and swallowing therapy. When using the device on the head or neck make sure only to work on low intensities.

## 6. CONTRAINDICATIONS

Treatments with the NOVAFON vibration sound wave devices are not permitted for patients with:

- Open wounds/eczema (in the body region to be treated)
- Arteriosclerosis (in the body region being treated)
- Current episodes of epilepsy
- Pregnancy
- Implants (in the body region being treated)
- Heart pacemakers, defibrillators and similar
- Thromboses (in the body region being treated)
- Brain pacemakers and deep brain stimulators
- Cardiac arrhythmia
- Tumours, cancerous and not cancerous
- Acute inflammations (in the body region being treated)
- Acute episodes of inflammatory diseases

Side effects:

Please note that in rare cases the following side effects may occur:

- Increased pain/unpleasant sensation
- Nausea/dizziness
- Skin irritation
- Reflex/involuntary muscle contractions

Please discontinue the treatment immediately, if you experience any of these side effects. If symptoms persist, please consult a doctor.

NOTE: Pain treatment in particular can lead to an initial worsening of the symptoms before you notice any improvement. NOTE: Slight reddening and warming is not a cause for concern. This is a desired effect of local vibration therapy, which is caused by the increased blood flow.

DO NOT USE for treating pains or swellings of unknown origin. In that case consult a medical professional for clarification and diagnosis.

## 7. TECHNICAL SPECIFICATIONS

### 7.1. SYMBOLS USED

|                                                                                   |                                                                                                        |
|-----------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------|
| ~                                                                                 | Alternating current (AC)                                                                               |
| —                                                                                 | Direct current (DC)                                                                                    |
|    | Device with protection class II according to DIN EN 61140                                              |
| Hz                                                                                | Hertz                                                                                                  |
| KB                                                                                | Short-term operation                                                                                   |
| mA                                                                                | Milliamperes                                                                                           |
| mAh                                                                               | Milliamperes per hour                                                                                  |
| A                                                                                 | Amp                                                                                                    |
| V                                                                                 | Volt                                                                                                   |
|    | Disposal according to the EU Directive on Waste Electrical and Electronic Equipment (WEEE)             |
|    | Displays the manufacturer of the medical device                                                        |
|    | Displays the serial number as defined by the manufacturer to identify a specific device.               |
|    | Medical-device                                                                                         |
|    | Unique-device-identification for medical products                                                      |
|    | Protection class of the application part: type BF                                                      |
|    | Temperature limit: Refers to the temperature limits to which the medical device can be safely exposed. |
|    | Humidity limit                                                                                         |
|    | Atmospheric pressure limit                                                                             |
|    | Refer to the instructions                                                                              |
|    | Use inside only                                                                                        |
|    | General warning symbols                                                                                |
|    | Intensität in 5 Stufen                                                                                 |
|    | UP                                                                                                     |
|    | DOWN                                                                                                   |
|    | Stand-by/ON/100 Hz / 75 Hz / 50 Hz/OFF                                                                 |
|    | Bluetooth                                                                                              |
|    | Battery full                                                                                           |
|   | Battery low                                                                                            |
|  | charging Battery                                                                                       |
|  | overheating protection active                                                                          |

### 7.2. TECHNICAL SPECIFICATIONS

- Operating conditions: 5 °C [41 °F] to 30 °C [86 °F]; 15 % to 93 % relative humidity, non-condensing; 700 hPa to 1060 hPa air pressure

- Storage/transport: -5 °C [23 °F] to 40 °C [104 °F]; 15 % to 93 % relative humidity, non-condensing; 500 hPa to 1060 hPa air pressure
- Operating mode: Short-term operation 20/15 min
- Operating noise: maximum 70 dBA
- Operating temperature: maximum 50 °C [122 °F] for maximum service life.
- Power plug Model 1180004-1P:
- Input power: AC 100–240 V, 50–60Hz, 300 mA
- Output power: DC 5 V, 2 A
- IP65 Protection class stating: Protected against dust in harmful quantities. Protection against water jets (nozzle) from any angle.
- LOT: Shows the date of manufacture of the power supply
- A type F (EU plug) power plug is included in the standard scope of delivery. Additional mains adapters with different power plugs are optionally available or already included in the corresponding country sets.
- USB-A charging cable to spring contact pin, cable length: approx. 1 m
- Battery: Built-in lithium-ion battery type 21700 with protection board, life span: 800 charging cycles. One charge of a new battery guarantees at least 6, 20-minute treatment sessions at medium intensity. Note that frequent use at the highest intensity level can decrease battery life.
- Battery performance: Capacity 5000 mAh, voltage 3,6 V, 18Wh
- Devices comply with IEC 60601-1-2:2014/AMD1:2020 and/or CISPR 11 Emission Class B and Group 1. Devices have been tested for home use in accordance with interference immunity testing levels.
- Product category: Class IIa medical device pursuant to the EU Regulation on Medical Devices (2017/745/EU), conformity with 2015/863/EU is also declared.
- Bluetooth (BLE): frequency 2,4–2,483 GHz, Modulation DSSS, modulation type GFSK 1000kbps, effective radiated power <0.15 mW at 25 mm, Contains FCC ID: 2AC7Z-ESP-MINI and IC: 21098-ESP32MINI1
- This device complies with part 15 of the FCC Rules.

- Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.
- This product complies with UK Radio Equipment Regulation 2017 and UK Medical Devices Regulations 2002
- Manufacturer: NOVAFON Elektromedizinische Geräte GmbH (NOVAFON GmbH), Daimlerstraße 13, 71384 Weinstadt, Germany

This NOVAFON device type uses RF energy for its internal function. Therefore, its RF emissions are very low and not likely to cause any interference in nearby electronic equipment. The device type is suitable for use in all establishments, including domestic and those directly connected to the public low-voltage supply network that supplies buildings used for domestic purposes. The device may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures, such as re-orientating or re-locating or shielding the location

The device complies with the following immunity test levels

| Immunity Test                                                 | Immunity test level                                                                                                                     |
|---------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------|
| Electrostatic Discharges immunity test (ESD)<br>IEC 61000-4-2 | Contact direct discharge $\pm 8$ kV<br>Contact indirect discharge $\pm 8$ kV<br>Air discharge $\pm 2$ kV $\pm 4$ kV $\pm 8$ kV $\pm 15$ |
| Power frequency magnetic field immunity test<br>IEC 61000-4-8 | 30 A/m (50Hz)                                                                                                                           |

|                                                                   |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                         |
|-------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Radiated RF EM Fields and Proximity Wire-less field IEC 61000-4-3 | <ul style="list-style-type: none"> <li>· 10V/m from 80 MHz to 2.7 GHz 80 % AM at 1kHz</li> <li>· 27 V/m, 18 Hz PM 50%, 385 MHz</li> <li>· 28 V/m, 18 Hz PM 50%, 450 MHz</li> <li>· 9 V/m, 217 Hz PM 50%, 710 MHz, 745 MHz, 780 MHz</li> <li>· 28 V/m, 18 Hz PM 50%, 810 MHz, 870 MHz, 930 MHz</li> <li>· 28 V/m, 217 Hz PM 50%, 1,720 GHz, 1,845 GHz, 1,970 GHz</li> <li>· 28 V/m, 217 Hz PM 50%, 2,450 GHz</li> <li>· 9 V/m, 217 Hz PM 50%, 5,240 GHz, 5,500 GHz, 5,785 GHz</li> </ul> |
| RADIATED FIELDS IN CLOSE PROXIMITY IMMUNITY TEST IEC 61000-4-39   | <ul style="list-style-type: none"> <li>· 30KHZ CW 8 A/M ; 134.2KHZ PULSE MODULATION 2.1KHZ 65 A/M</li> <li>· 13.56 MHZ PULSE MODULATION 50KHZ 7.5 A/M</li> </ul>                                                                                                                                                                                                                                                                                                                        |

## 8. IMPORTANT SAFETY INSTRUCTIONS

Before starting to use the device, please read the user manual and these safety instructions carefully and strictly follow them.

### 8.1. WARNINGS

When using an electrical device basic precautions should always be followed including the following to reduce the risk of burns, fires, electric shock, possibly severe injury and possible death to the user or other people:

- ⚠ WARNING: Do not use the NOVAFON device during charging.
- ⚠ WARNING: Always store the device in a dry and clean location.

- ⚠ WARNING: Before connecting the power plug to the mains power source, check that the mains voltage corresponds to the voltage specification on the power plug. Connect the NOVAFON device only to a power outlet that delivers the correct voltage.
- ⚠ WARNING: To charge the device, use only the charging cable and power supply included in the delivery or the separately available charging stand. Use the correct connector plug, if applicable. Non-observance can lead to increased electromagnetic emissions or reduced electromagnetic immunity and thus to malfunctions or unexpected operational behaviour.
- ⚠ WARNING: Always unplug the device and all charging accessories from the electrical outlet after charging and before cleaning. Make sure that the mains plug is easily accessible at all times so that you can easily disconnect the device from the mains in an emergency. Do not place heavy objects or furniture in front of the power adapter, even when using the charging stand.
- ⚠ WARNING: Ensure that devices are completely dry and with no cleaning agent, disinfectant or oil/cream residues before using the charging stand or charging cable.
- ⚠ WARNING: Do not bring the device and charging accessories (plug, charging cable, charging stand) into contact with hot surfaces [e.g. heater, hotplate, etc.]. The housing of the device or adapter or the insulation of the cable could be damaged.
- ⚠ WARNING: Do not use the device when taking a shower or bath.
- ⚠ WARNING: Do not use the device where aerosol (spray) products are used, in other explosive atmospheres, or anywhere where oxygen is administered.
- ⚠ WARNING: Do not place or drop the device into water or other liquids.
- ⚠ WARNING: Do not reach for a device that has fallen into water, especially when the charging cable is connected to a power outlet. Unplug immediately.
- ⚠ WARNING: Do not place or store the device where it could fall or be pulled into a tub, pool or sink.
- ⚠ WARNING: Protect the device, charger and charging stand from drop impact and shocks.
- ⚠ WARNING: This device may not be modified without the manufacturer's permission.
- ⚠ WARNING: NOVAFON devices may only be used with original NOVAFON accessories. The use of accessories from other manufacturers can impair the function and performance of NOVAFON devices. This also applies to the power adapter and charging cables.
- ⚠ WARNING: Do not use the device and either discard or return to your dealer for examination if any of the following conditions occur:
  - if the device fails to work correctly
  - if the device is damaged, e.g. after having been dropped
  - if the device was immersed in water or other liquids
  - if the device exhibits other safety-related problems, such as emitting smoke, unusual or excessive heat etc.
- ⚠ WARNING: In order to prevent adverse events to the patient and operator due to electromagnetic disturbances, the NOVAFON shall not be operated outside its intended EMC environment. Furthermore, the NOVAFON shall not be operated if the enclosure, cables or measures for electromagnetic shielding are damaged.
- ⚠ WARNING: Do not use the power plug, charging cable or
  - charging stand:
    - in the event of visible damage or after dropping the power plug, charging cable or charging stand
    - if the power plug, charging cable or charging stand has been exposed to extreme moisture
    - if liquid has penetrated the unit
- ⚠ WARNING: Never charge the device for more than 24 hours at a time. Do not leave the device unattended. Disconnect the device from the power supply at night. This also applies when using the charging stand.
- ⚠ WARNING: Do not place the device on flammable surfaces when charging.
- ⚠ WARNING: Never leave the device unattended when it is turned on.
- ⚠ WARNING: Do not clamp or pinch any part of your body, especially your fingers, between the attachment and the device. During the treatment, hold the device only by the

device handle.

- ⚠ WARNING: Never drop or insert any foreign object into any opening on this device. Do not insert any objects (including wires) into openings in the device.
- ⚠ WARNING: Wireless communication devices (such as Wi-Fi stations, mobile phones, cordless phones, and related base stations) can affect the device and should be kept at least 0.5 m (2 feet) away from the device during use. This also applies to cables and plug-in power supplies. Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the Manufacturer. Otherwise, degradation of the performance of this equipment could result.
- ⚠ WARNING: Careful supervision is necessary when used on or by children, immobile or disabled individuals. NOVAFON devices should only be used by users from the age of 18 years who have read the instructions and are competent to use the device safely.
- ⚠ WARNING: Do not use on sleeping or unconscious individuals.
- ⚠ WARNING: Keep the device out of reach of children when not in use.
- ⚠ WARNING: Small parts (e.g. cover caps) could come loose and be swallowed. Risk of suffocation!
- ⚠ WARNING: Keep children away from packaging materials. There is a risk of suffocation.
- ⚠ WARNING: Do not use on genitals, this device is not intended as an intimate pleasure toy.
- ⚠ WARNING: Do not use outdoors or in wet or damp areas.
- ⚠ WARNING: Never operate the power plug or charging stand in a high-humidity environment or in the presence of moisture. Please note that moisture is also generated by e.g. nebulisers and paraffin baths.
- ⚠ WARNING: Do not use or charge the device under blankets, pillows, towels, clothing, etc. or in places where ventilation is restricted, as it may overheat and could cause fire, electric shock or other personal injury.

- ⚠ WARNING: Use the device only for its intended use as described in this manual.
- ⚠ WARNING: Avoid contact with eyes.
- ⚠ WARNING: Avoid contact with mucous membranes unless using specially designed attachments.
- ⚠ WARNING: Do not use the device together with other medical devices and do not stack the devices on such products, as this may affect the performance and function of the device as well as the other product.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

Consult the dealer or an experienced radio/TV technician for help.

- ⚠ WARNING: Avoid overheating the device. Do not use the device for more than 20 minutes per application. Allow the device to cool down for at least 15 minutes before switching it on again.
- ⚠ WARNING: For technical reasons, the device can reach a temperature of up to 50 °C, especially on the head. Therefore, do not hold the device by your head for long periods of time. Change your grip position if the device feels uncomfortably warm. Switch off the device if any part of it feels hot and allow it to cool down thoroughly

before using it again.

## 8.2. OTHER PRECAUTIONS

NOTE: Use the initial session to find out how your body responds to vibration therapy. If you experience any discomfort, dizziness or nausea, stop the session immediately.

NOTE: Do not use the device if any of the contraindications described in chapter 6 apply to you.

NOTE: Do not use the device for more than 20 minutes per application. Leave it to cool down for at least 15 minutes before restarting.

NOTE: Make sure to secure the attachments fully onto the NOVAFON device and keep it on during treatment.

NOTE: If you perform multiple treatment sessions in quick succession with a maximum duration of 20 minutes and only use a single attachment, the temperature of the attachment can reach up to 50 °C. You can avoid this by removing the attachment between sessions (e.g. for cleaning) or by using alternate attachments within the session. Stop the treatment if the contact surface feels uncomfortably warm.

NOTE: Make sure that no substances, such as liquids, creams or the like, can penetrate into the device or the power plug.

NOTE: To prevent damage, make sure that the device and charging stand are standing or resting on a stable surface during the charging process.

NOTE: Do not twist the device in the charging stand. This may cause damage.

NOTE: Keep the device away from dust, lint and other particles. The penetration of dust, lint or other particles can cause the device to fail. Clean your device regularly with suitable agents to prevent any damage caused by contamination. Please note chapter 9.1.

NOTE: To prevent premature wear, only store the device in a dry and clean place that is protected against UV light (incl. sunlight).

NOTE: Use only non-alcoholic cleaning supplies. Alcohol can attack the surfaces of the housing and the attachments.

Do not clean the device during charging. Disconnect the device from the power supply when cleaning. If used privately, clean the housing and attachments using a clean cloth moistened with fresh water or a mild soapy solution. For clinical and professional use, clean and disinfect device housings and all attachments with an appropriate surface disinfectant suitable for medical devices before and after each use. As a general rule, only use alcohol-free disinfectants that are suitable for use on sensitive surfaces, such as NOVACLEANER by NOVAFON, SinAlc sensitive by HIMED or RHEOSEPT-WD plus wipes by Rheosol. If necessary, dry off with a soft cloth.

If you are using the charging stand, this can be cleaned with a clean cloth moistened with fresh water or a mild soapy solution if necessary. For professional use: To prevent contamination, only place cleaned devices in the charging stand. Ensure that devices are completely dry and with no cleaning agent, disinfectant or oil/cream residues before using the charging stand or charging cable.

## 9.2. TROUBLESHOOTING

If your device automatically switches to a lower setting, or you are unable to increase the intensity, this is probably due to a low battery level. In this case, you should charge your device.

You can also use the FAQs section of our website to find helpful answers to frequently asked questions.

If other malfunctions should occur in the device, please contact our customer service department. Never open the device yourself to rectify faults. Any attempt to repair the device yourself will void your warranty. For this product, the manufacturer declares sole responsibility for conformity with the requirements for class IIa medical devices pursuant to the EU Regulation on Medical Devices (2017/745/EU). Please contact us if you suspect that the device has caused a serious incident. As a user, you have the right to report incidents of this type to the relevant national authority (e.g. Ministry of Health).

## 9. MAINTENANCE AND REPAIR

### 9.1. CLEANING AND DISINFECTION

#### 9.2.1 FACTORY SETTINGS

You can reset your device to factory settings by plugging the power off device into the charging cable and then pressing

and holding the ON/OFF button and the intensity button at the top for about 10-3 seconds at the same time. The "Reset" indicator appears on the display. Continue to press and hold the button until the ring closes completely.

### 9.2.2 FORCED DEVICE RESTART (HARD RESET)

If your device no longer responds to any measures, including the factory reset described above, you can force a device restart by holding down the lower intensity button for approx. 7 seconds with the cable connected. The device will then start at 100 Hz and the lowest intensity.

### 9.3. SAFETY CHECK, MAINTENANCE AND REPAIR

The NOVAFON device does not require regular inspection or maintenance. To ensure safe operation, however, you should check the device regularly for obvious damage. We also recommend that you carry out an electrical safety test, e.g. compliant with DGUV V3 (German Social Accident Insurance), at least every 2 years. Please note that there may be national laws that oblige you to carry out safety checks on the device. This does not generally apply to private individuals. If you have any problems with your appliance either discard or return it to your dealer. Attachments shall be replaced once they show clear signs of wear or damage (cracks, deformations, change in surface). Information regarding repair options and spare parts can be found on our website.

NOTE: Attachments and batteries are parts subject to wear and as such are subject to a limited warranty. The battery in your device can be replaced by NOVAFON or, where applicable, a technician authorised by NOVAFON.

### 9.4. FIRMWARE UPDATES

Occasionally it may be necessary to update the firmware on your device. You will be informed of this either in the app or on our website. To carry out an update, your device must be paired with the app (see chapter 4.5), the app will guide you through the necessary steps. The status of the update is displayed on the screen while it is being carried out.

### 9.5. DISPOSAL

Devices must never be disposed of with common household waste. For environmentally sound disposal, the user is obli-

gated to dispose of the old device at a designated collection point for the disposal of electrical and electronic equipment. The device contains a permanently installed battery and must be disposed of accordingly. Please observe the national legal regulations when disposing of the device. You can also send your old device to NOVAFON; we will gladly dispose of it for you. This device is subject to EU Directive on Waste Electrical and Electronic Equipment (WEEE).

To ensure the conservation of resources, please dispose of the packaging separately as waste paper.

## 10. WARRANTY

From the date of delivery, we grant a warranty of 2 years (24 months) for defects resulting from material or production defects. Warranty coverage is subject to the presentation of a proof of purchase. The battery, charging cable, power plug and the attachments has a limited warranty of 6 months. Excluded from the warranty: Damage due to improper use, normal wear and tear and consumption (also valid for accessories).

The right of rescission (reversal of the purchase), reduction (discount on the purchase price) or subsequent delivery is initially excluded. However, you have a claim to reduction or conversion at your discretion if we do not remedy a defect that is subject to our warranty within a reasonable period of time. Any damage caused by failure to use the product in accordance with its intended use or resulting from improper or unreasonable use or maintenance is not covered by the warranty. Such damage is the sole responsibility of the user. Failure to comply with this instruction manual will void the warranty.

Some states may not allow such limitation, so it may not apply to you. This limited warranty gives you specific and you may have other rights which may vary from state to state. Get in touch with your dealer or supplier to claim warranty.

## 11. CUSTOMER SERVICE

The device may only be repaired by NOVAFON Customer Service. Please do not send in your defective device without being asked, but rather clarify the damage and repair costs

with us first. Repair could be more expensive than a new device. If the repair is no longer financially feasible within the warranty period, NOVAFON offers a replacement unit (again with a 24-month warranty).

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- <sup>1</sup> Lundeberg T. 1983. Vibratory stimulation for the alleviation of chronic pain. Lundeberg T, Nordemar R, Ottoson D. 1984a. Pain alleviation by vibratory stimulation. Lundeberg T. 1984b. Long-term results of vibratory stimulation as a pain relieving measure for chronic pain. Lundeberg T. 1984c. The pain suppressive effect of vibratory stimulation and transcutaneous electrical nerve stimulation (TENS) as compared to aspirin. Lundeberg T. 1985b. Relief of pain from a phantom limb by peripheral stimulation. Lundeberg T. et al. 1987. Vibratory stimulation compared to placebo in alleviation of pain. Staud R, Robinson ME, Goldman CT, Price DD. 2011. Attenuation of Experimental Pain by Vibro-Tactile Stimulation in Patients with Chronic Local or Widespread Musculoskeletal Pain. Serritella et al. 2020. Local Vibratory Stimulation for Temporomandibular Disorder Myofascial Pain Treatment: A Randomised, Double-Blind, Placebo-Controlled Preliminary Study. Pain Research Management. Beinert K, Keller M, Taube W. 2014. Neck muscle vibration can improve sensorimotor function in patients with neck pain. Beinert K, Englert V, Taube W. 2018. After-effects of neck muscle vibration on sensorimotor function and pain in neck pain patients and healthy controls – a case-control study.
- <sup>2</sup> Murillo et al 2014. Focal vibration in neurorehabilitation. Eur J Phys Rehabil Med. 50(2):231–42. Caliendo et al. 2012. Focal muscle vibration in the treatment of upper limb spasticity: a pilot randomized controlled trial in patients with chronic stroke. Arch Phys Med Rehabil. 93(9):1656–61. Casale et al. 2014. Localized 100 Hz vibration improves function and reduces upper limb spasticity: a double-blind controlled study. European Journal of Physical and Rehabilitation Medicine. Eur J Phys Rehabil Med. 50(5):495–504. Noma et al. 2012. Anti-spastic effects of the direct application of vibratory stimuli to the spastic muscles of hemiplegic limbs in post-stroke patients: a proof-of-principle study. J Rehabil Med. 44(4):325–30. Mandic et al. 2012. Kinematic analysis of upper-extremity movements after segmental muscle vibration therapy. Gait & Posture. 35: S21–S22. Liepert & Binder. 2010. Vibration-induced effects in stroke patients with spastic hemiparesis – a pilot study. Restor Neurol Neurosci. 28(6):729–35. Paoloni et al. 2014. Segmental muscle vibration modifies muscle activation during reaching in chronic stroke: A pilot study. NeuroRehabilitation. 35(3):405–14. Marconi et al. 2011. Long-term effects on cortical excitability and motor recovery induced by repeated muscle vibration in chronic stroke patients. Neurorehabil Neural Repair. Jan; 25(1):48–60. Tavernese et al. 2013. Segmental muscle vibration improves reaching movement in patients with chronic stroke. A randomized controlled trial. NeuroRehabilitation. 32(3):591–9. Celletti & Camerota. 2011. Preliminary evidence of focal muscle vibration effects on spasticity due to cerebral palsy in a small sample of Italian children. Clin Ter. 162(5):125–8. Mortaza N. et al. 2019. Upper limb tendon/muscle vibration in persons with subacute and chronic stroke: a systematic review and meta-analysis. Alashram AR et al. 2019. Effectiveness of focal muscle vibration on hemiplegic upper extremity spasticity in individuals with stroke: A systematic review.
- <sup>3</sup> Schindler et al. 2002. Neck muscle vibration induces lasting recovery in spatial neglect. Johannsen et al. 2003. Lasting amelioration of spatial neglect by treatment with neck muscle vibration even without concurrent training. Kamada et al. 2011. Effects of 5 minutes of neck-muscle vibration immediately before occupational therapy on unilateral spatial neglect. Karnath et al. 2002. The perception of body orientation after neck-proprioceptive stimulation. Effects of time and of visual cueing.
- <sup>4</sup> Barsties v. Latoszek. 2020a. Preliminary study of Novafon Local Vibration Voice Therapy for dysphonia treatment. Barsties v. Latoszek. 2020b. Treatment effectiveness of Novafon Local Vibration Voice Therapy for Dysphonia Treatment. Schuster et al. 2015. Finally back to voice therapy options in dysphonia. Yiu. 2021. Vibrational Therapies for Vocal Fatigue. Hricová M, Hammer S. 2020. Lokale Vibrationstherapie in der Neurorehabilitation – Eignet sich der intraorale Vibrationsspatel zur Verbesserung der Zungenkraft bei Schlaganfallpatienten? – Eine Pilotstudie. Grant. 1982. The use of a manual vibrator in the speech therapy program of four school-age mentally retarded children.
- <sup>5</sup> Eklund G, Steen M. 1969. Muscle vibration therapy in children with cerebral palsy. (Inapplicable)
- <sup>6</sup> Koschorke GM, Meyer RA, Tillman DB, Campbell JN. Ectopic excitability of injured nerves in monkey: entrained responses to vibratory stimuli. J Neurophysiol. 1991 Mar;65(3):693–701. doi: 10.1152/jn.1991.65.3.693. PMID: 1711106.

More literature can be found on our website. Other languages are available online.