M E D I C A L

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- 1. PREFACE**
- 2. PRODUCT DESCRIPTION**
  - 2.1. DEVICE TYPES
    - 2.1.1. NOVAFON pro
    - 2.1.2. NOVAFON power
    - 2.1.3. NOVAFON classic
  - 2.2. ACCESSORIES
    - 2.2.1. Disk attachment (for pro)
    - 2.2.2. Disk attachment (for power, classic)
    - 2.2.3. Ball attachment
- 3. OPERATION**
  - 3.1. DEVICE START-UP
  - 3.2. INTENSITY CONTROL
  - 3.3. ATTACHMENTS
- 4. APPLICATIONS & USE**
  - 4.1. SELF-APPLICATION AT HOME
  - 4.2. PROFESSIONAL APPLICATION IN NEUROREHABILITATION
    - 4.2.1. Spasticity
    - 4.2.2. Neglect
- 5. CONTRAINDICATIONS**
- 6. TECHNICAL SPECIFICATIONS**
  - 6.1. SYMBOLS USED
  - 6.2. TECHNICAL SPECIFICATIONS
- 7. SAFETY INSTRUCTIONS**
- 8. SERVICE & MAINTENANCE**
  - 8.1. CLEANING & DISINFECTION
  - 8.2. TROUBLESHOOTING
  - 8.3. SAFETY CHECKS
  - 8.4. DISPOSAL
- 9. WARRANTY**
- 10. CUSTOMER SERVICE**

## 1. PREFACE

Congratulations on the purchase of an original NOVAFON sound wave device. Our therapy devices have delivered excellent performance and reliability for decades. Please read the following pages carefully to ensure safe, effective and long-term use of your NOVAFON sound wave device. The description of the foot-notes can be found on the last page of the booklet.

## 2. PRODUCT DESCRIPTION

NOVAFON sound wave devices are designed for local vibration therapy. Various attachments are available to apply the mechanical vibrations to the tissue. NOVAFON sound wave devices are primarily used in pain therapy and neurorehabilitation. Owing to the safe and simple use of our devices, local vibration therapy is ideal for a variety of applications. NOVAFON sound wave devices are available with one frequency level (100 Hz) or with two levels (50 Hz / 100 Hz). The devices are designed both for professional use and for home use.

### 2.1. DEVICE TYPES

The individual device types differ in terms of the frequency settings, intensity, alloy type of the device housing (chrome-plated, gold-plated or plastic) and housing shape and with respect to the accessories they include (with or without extension handle).

**2.1.1. NOVAFON pro** The versatile device is suitable for use in speech therapy and occupational therapy, as well as for home use. Thanks to the fold-out extension bar, hard-to-reach areas such as the feet or back can be treated particularly well in self-use. The device can be used with its gentle vibrations (either 50 or 100 Hz) even on sensitive body parts easily.

**2.1.2. NOVAFON power** Due to the more intense vibration strength, the NOVAFON power is particularly suitable for the treatment of larger muscle groups, e.g. in orthopaedic disorders. The generated vibrations (optionally 50 or 100 Hz) are stronger due to the higher vibration amplitude and penetrate deeper into the tissue. The power device is particularly suitable for sports and physiotherapy as well as medical practice.

**2.1.3. NOVAFON classic** The basic model enables uncomplicated operation and easy treatment of common indications. It has only one setting (100 Hz) and therefore facilitates easy handling. The NOVAFON classic can be used both in the practice and at home.

	NOVAFON power	NOVAFON pro	NOVAFON classic
<b>Frequency levels</b>	two	two	one
Level			
Normal operation	Level 1 100 vibrations per second (Hz)	Level 1 100 vibrations per second (Hz)	Level 1 100 vibrations per second (Hz)
Level			
Percussion	Level 2 50 vibrations per second (Hz)	Level 2 50 vibrations per second (Hz)	not available
Off position			
<b>Intensity</b>	+++++	+++++	+++++
<b>Effective depth</b>	6 cm +	6 cm	6 cm
<b>Intensity control</b>			
<b>Features/ Inclusions</b>		with extension handle	

## 2.2. ACCESSORIES

All NOVAFON sound wave devices come standard with two exchangeable attachments. NOVAFON GmbH company offers additional attachments, which are not included in the scope of supply. Please contact your retailer for further information or go to [www.novafon.com](http://www.novafon.com). NOVAFON devices are only to be used with original NOVAFON accessories. The use of accessories from other manufacturers can impair the function and performance of the NOVAFON devices.

**2.2.1. Disk attachment (for NOVAFON pro)** The disk attachment is designed for the treatment of larger body regions that can be easily treated by using the device in a sweeping motion. The disk attachment is included in the delivery. It can also be ordered as a replacement part.

**2.2.2. Disk attachment (for NOVAFON power, NOVAFON classic)** The disk attachment is designed for the treatment of larger body regions that can be easily treated by using the device in a sweeping motion. The disk attachment is included in the delivery. It can also be ordered as a replacement part.

**2.2.3. Ball attachment** The round head attachment can be used for concentrated treatment of smaller areas. It is included and also available as a spare part.

### **3. OPERATION**

#### **3.1. DEVICE START-UP**

Plug the power plug into a power outlet that provides the specified voltage. Ensure that the cable is not stretched to a length of over 3 metres as the cable might break. Use an extension cord, if required. Turn on the NOVAFON device using the flip switch . This flip switch is also used to select the frequency level in the NOVAFON pro and power models .

#### **3.2. INTENSITY CONTROL**

The intensity can be adjusted by turning the control knob . Turning the control knob clockwise (viewed from the top) reduces the intensity. Consequently, turning the control knob counter-clockwise increases the intensity. Please bear in mind that you need to turn the control knob several times to reach the highest or lowest intensity. We recommend that you start at a low intensity and then increase it as required and subject to individual sensitivity levels. Do not apply any pressure when using the NOVAFON device. Instead of enhancing treatment effectiveness, this would interfere with the vibratory performance. Perfect vibrations are generated by the mere contact of the NOVAFON device with the skin.

#### **3.3. ATTACHMENTS**

The disk attachment and ball attachment attachments can be easily replaced. Unscrew the attachment counter-clockwise to remove it or turn it clockwise to tighten it.

### **4. APPLICATIONS & USE**

The local vibration therapy with the NOVAFON sound wave device can be used as a supporting measure in a number of diseases of the musculoskeletal and the nervous system. For example, NOVAFON sound wave devices are used by physicians and therapists for the treatment of pain in the musculoskeletal system (e.g. muscular or joint pain) and in the rehabilitation of neurological diseases (e.g. spasticity or neglect after stroke). Due to their simple handling, the devices can also be used in the home environment. Please note that the NOVAFON sound wave devices can only alleviate symptoms but never actually cure the underlying disease, nor replace a doctor's visit. Despite being safe, simple and non-invasive, we recommend that you discuss the treatment with your treating physician or therapist. If symptoms get worse, seek medical advice. Discontinue the treatment immediately, if you experience nausea, dizziness, increased pain, or skin irritation. Please always pay attention to the contraindications listed in chapter 5.

#### **4.1. SELF-APPLICATION AT HOME**

Pain is a common symptom of both acute and chronic conditions, such as epicondylitis, tendinitis, myalgia, or fibromyalgia, which can be reduced by local vibration therapy.<sup>1</sup> Taking into account the contraindications (chapter 5), the NOVAFON is guided on or around the treatment area in a straight line, selectively or in slow circular movements. The NOVAFON can also be applied at trigger

points near the painful area. In case of uncertainty, please consult your treating physician or therapist regarding your individual application. The NOVAFON can be used directly on the skin or through fabric/clothing. During the application, make sure that the sound wave device glides gently and without pressure over the body area. Use the entire surface of the attachment. This is the only way the penetration depth effect of the NOVAFON can be fully deployed and the vibrations are transmitted optimally. Use only the NOVAFON's own weight for a comfortable application.

The therapeutic application should not exceed a maximum of two 20-minute treatment units per day. Please note that the NOVAFON must be switched off for 15 minutes after using it for 20 minutes in order to avoid overheating. When applied to sensitive areas (e.g. face, neck, head) or severely painful body parts, the duration of application should be reduced and adjusted to accommodate the individual needs.

#### NOTE

Application of liniments (medicated topical preparations): Generally, only apply small amounts of creams, ointments, cosmetic or medicated liniments on the skin or place a fine cloth between the device and the skin. Do not allow liquids or residues to penetrate the NOVAFON so as not to damage it.

### 4.2. PROFESSIONAL APPLICATION IN NEUROREHABILITATION

Note: The use of local vibration therapy for neurological diseases should only be carried out by specialists (therapists, physicians, nurses) or after detailed instruction by such a person.

**4.2.1. Spasticity** Spasticity is a common sequence of stroke and other disorders of the central and peripheral nervous system. Local vibration therapy, combined with physiotherapy, provides a simple and drug-free treatment method. A variety of studies demonstrated a reduction in spasticity as well as improved motor control and function.<sup>2</sup> The mechanisms of effectiveness are the tonic vibration reflex and the excitation of the sensorimotor cortex. Taking into account the contraindications (chapter 5), the spastic muscle itself (flexors) is treated with 50 Hz (level 2) and the antagonist of the spastic muscle (extensors) with 100 Hz (level 1). The starting point can be directly on the tendon or on the muscle. It may be treated locally, i.e. focus on one point, or in sweeping movements. The application with the NOVAFON sound wave device should not exceed 20 minutes per unit. The treatment can be performed up to 5 times a week. Please keep in mind that using the appliance incorrectly can lead to increased muscle tone. Stop the treatment immediately if any side effects occur (see section 5).

**4.2.2. Neglect** Several studies have shown an improvement in the visual exploration performance in stroke patients with neglect when the neck muscles, contralateral to the lesion, were stimulated by vibration.<sup>3</sup> This effect stems from the fact that the stretch receptors in the muscles and tendons of the neck are stimulated by vibration, resulting in an apparent rotation of the head. To carry

out the therapy, the corresponding stimulation point must be found on the contra-lesional, posterior neck muscles, taking into account the contraindications (chapter 5). This point is determined by fixing a bright spot on the wall (e.g. by means of a laser pointer) in a darkened room. The therapist slowly guides the NOVAFON sound wave device along the neck muscles at medium intensity. The patient sits on a chair for the treatment. The correct stimulation point is determined as soon as the patient reports that the fixed point is moving without moving his/her head. The total treatment time should not exceed 40 minutes, whereby the pure stimulation time should be interrupted by a 5 minute break after 10 minutes. A total of at least 15 treatment sessions are necessary to achieve a lasting effect. Stop the treatment immediately if any side effects occur (see section 5).

## **5. CONTRAINDICATIONS**

Do not perform any treatment with the NOVAFON sound wave devices if the following contraindications apply:

- open wounds / eczema (in the body region requiring treatment)
- arteriosclerosis (in the body region requiring treatment)
- recent episodes of epilepsy
- pregnancy
- implants (in the body region requiring treatment)
- pacemakers
- thrombosis (in the body region requiring treatment)
- deep-brain stimulators
- cardiac arrhythmia
- tumours
- acute inflammations (in the treated region of the body)
- acute episodes of inflammation-related diseases

Side effects:

Please note that in rare cases the following side effects may occur:

- Increased pain / unpleasant sensation
- Nausea / dizziness
- Skin irritation
- Reflex / involuntary muscle contractions

Please stop the treatment immediately, should any of these side effects occur.

Please note that an initial worsening of the symptoms is especially possible before an improvement is noticeable when treating pain. Please note that slight reddening and warming of the skin is not a cause for concern. This is a desired effect of local vibration therapy, which is caused by the increased blood flow.

## 6. TECHNICAL SPECIFICATIONS

### 6.1. SYMBOLS USED

- ~ Alternating current
-  Device of protection class II according to DIN EN 61140  
IP44 (splash-proof)
- Hz** Hertz
- KB** Short-time duty
- mA** Milliampere
- V** Volt
-  Disposal according to the EU Directive on Waste Electrical and Electronic Equipment (WEEE)
-  Displays the manufacturer of the medical device according to the EU Directives 93/42/EEC.
-  Displays the serial number of the manufacturer so that a specific medical device is identified
-  Type BF: A type BF (Body Floating, conductive contact with the body) applied part/device is connected to the patient's body to transmit electrical energy or an electrophysiological signal to or from the body
-  Temperature limit: refers to the temperature limits to which the medical device can be safely exposed.
-  Humidity limitation
-  Atmospheric pressure limitation
-  Refer to instruction manual/booklet
-  General warning sign

### 6.2. TECHNICAL SPECIFICATIONS

- Operating conditions: 5 °C to 30 °C; 15% to 93% relative humidity, not condensing; 700 hPa to 1060 hPa atmospheric pressure
- Storage/transport: -25 °C to 70 °C; up to 93% relative humidity, not condensing; 700 hPa to 1060 hPa atmospheric pressure
- Nominal voltage: Primary 230V~, 50 Hz, 65mA; Secondary 17V~, 50Hz, 450mA
- Nominal frequency: 50 Hz
- Operating mode: KB 20/15 min
- Nominal consumption: 9.5W
- The devices are conform with EN 60601-1-2:2015 or CISPR 11 Emission Class A and Group 1 respectively. The devices have been tested using the interference immunity test level for home use
- Product category: Medical device class IIa according to the Medical Devices Directive (93/42/EEC), conformity with 2011/65/EU is still declared.
- Manufacturer: NOVAFON GmbH, Daimlerstraße 13, 71384 Weinstadt
- Operating noise: maximum 70 dBA

## 7. SAFETY INSTRUCTIONS

Correct and safe operation of the NOVAFON sound wave device can only be ensured if NOVAFON accessories are used. Before starting to use the device, please read the user manual and these safety instructions carefully and strictly follow them.

- ⚠ WARNING: Do not modify this device without the manufacturer's prior consent.
- ⚠ WARNING: NOVAFON devices are only to be used with original NOVAFON accessories. The use of accessories from other manufacturers can impair the function and performance of the NOVAFON devices.

- Handle the device carefully.
- Always store the device in a dry and clean location.

Information: Before each use, check the device for any signs of damage or wear.

- ⚠ WARNING: Do not use the device and have it repaired by the customer service if any of the following conditions occur:
  - if the power cable or plug is damaged.
  - if the device fails to work correctly.
  - if the device is damaged.
  - if the device exhibits safety-related problems.

- Use an extension cord if the distance to the power outlet is over 3m.
- Before connecting the device to the power source, check that the mains voltage corresponds to the voltage specification on the transformer. Connect the device only to a power source that delivers the correct voltage.
- To disconnect the device, unplug the power plug from the power outlet. Never pull the cable.
- Never leave the device unattended when it is turned on.
- Do not use the device and transformer for over 20 minutes per application. Leave it to cool down for at least 15 minutes before you continue.
- Avoid any contact of the power cable with hot surfaces.

- ⚠ WARNING: Never use the device when taking a shower or bath, in an environment that has been enriched with oxygen or where there is a risk of explosion or in an explosive atmosphere.

- Never place the device into water or other liquids.
- Protect the device and transformer from impact and shocks.
- During treatment, only hold the handle or (extended) extension handle of the device.

- ⚠ WARNING: Wireless communication devices (e.g. WiFi stations, mobile phones, cordless phones and their base stations) may interfere with the sound wave devices and should be kept at a minimum distance of 0.5 m from the device during treatment. This is also valid for the cable and the plug-in power supply.

- ⚠ WARNING: Do not use the device at the same time as other medical devices as this might influence performance and functionality of either or both devices.

- Keep the device out of the reach of children.
- ⚠ **WARNING:** When using the device, bear in mind that the cable may cause a strangulation hazard.
- Do not squeeze or pinch any part of your body, especially your fingers, between the sound plate/ball and the device or the fold-out extension handle and the device.
  - Loosen the attachment from the device at regular intervals (twist off and twist on) as this ensures that the attachment cannot become stuck in the thread.
- ⚠ **WARNING:** Do not use under blankets, cushions, towels, etc. or in places where the device ventilation could become blocked. There is a risk of overheating that could result in a fire breaking out or people being injured otherwise.

## **8. SERVICE & MAINTENANCE**

### **8.1. CLEANING & DISINFECTION**

For cleaning, disconnect the device from the power circuit. For private use, use a clean damp cloth (with clean water or mild soapy water) to clean the unit casing and attachments. For clinical and professional use, clean and disinfect the instrument casing, disk attachment, and round head attachment with an appropriate surface disinfectant for medical devices before and after each use. For disinfection, NOVAFON generally recommends alcohol-free disinfectants that are suitable for use on sensitive surfaces, e.g. SinAlc sensitive by HIMED or RHEOSEPT-WD plus Wipes from Rheosol. If necessary, wipe dry with a soft cloth. Loosen the attachment from the device at regular intervals (twist off and twist on) as this ensures that the attachment cannot become stuck in the thread.

### **8.2. TROUBLESHOOTING**

Contact our Customer Service if any defects or failures occur. Never open the device yourself to correct any malfunction. Attempts to repair the device yourself will void the warranty.

### **8.3. SAFETY CHECKS**

Regularly scheduled inspection or maintenance is not provided for the NOVAFON device. You should nevertheless inspect the device for signs of obvious damage at regular intervals in order to ensure a safe use. An electrical safety inspection conform with DGUV 3 for example is also recommended at least every 2 years. Please note that national laws may still require you to perform security checks on the device. This usually does not apply to private individuals.

### **8.4. DISPOSAL**

The disposal of the NOVAFON device is subject to the requirements of the EU Directive on waste electrical and electronic equipment (WEEE Directive). It must not be disposed of in municipal waste. Users are required to use designated collection points when disposing of waste electrical and electronic equipment to minimize environmental impact. Please observe the national statutory

regulations for the disposal of the device. You are free to return your old device to NOVAFON. We will take care of its disposal for you.

## **9. WARRANTY**

The device is warranted against defects in materials or workmanship for a period of two (2) years from the date of delivery. Warranty coverage is subject to the presentation of a proof of purchase. Please contact your dealer directly for any warranty claims. The right of rescission (cancellation of purchase), the right to demand a reduction in the purchase price (discount) or the right to demand subsequent delivery are generally excluded. However, the buyer is entitled, at its own discretion, to demand a reduction in the purchase price or withdraw from the purchase if the seller fails to remedy a defect covered by the warranty within a reasonable period of time. Any damage caused by failure to use the device in accordance with its intended use is not covered by the warranty. Such damage is the sole responsibility of the user. The power cable must not be stretched to a length of over 3 metres. If a greater distance needs to be covered, an extension cord must be used. Otherwise the cable may break. Furthermore, we do not warrant for any damage resulting from improper use or failure to follow the operating instructions. Failure to comply with these instructions will void the warranty.

## **10. CUSTOMER SERVICE**

The NOVAFON device requires precise adjustment. Consequently, any necessary repairs must be performed exclusively by the NOVAFON Customer Service. Please do not send in your defective device without first contacting us by telephone to report the damage and determine the scope of the repair. The repair might be more expensive than a new device. If, during the warranty period, the repair of a device proves too costly, NOVAFON will offer a replacement device (to which a full 24-months' warranty period will apply).

This User Manual is also available in PDF format at [www.novafon.com](http://www.novafon.com).

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## NOTES

- <sup>1</sup> Lundeberg T. 1983. Vibratory stimulation for the alleviation of chronic pain. Lundeberg T, Nordemar R, Ottoson D. 1984a. Pain alleviation by vibratory stimulation. Lundeberg T. 1984b. Long-term results of vibratory stimulation as a pain relieving measure for chronic pain. Lundeberg T. 1984c. The pain suppressive effect of vibratory stimulation and transcutaneous electrical nerve stimulation (TENS) as compared to aspirin. Lundeberg T. 1985b. Relief of pain from a phantom limb by peripheral stimulation. Lundeberg T. et al. 1987. Vibratory stimulation compared to placebo in alleviation of pain. Staud R, Robinson ME, Goldman CT, Price DD. 2011. Attenuation of Experimental Pain by Vibro-Tactile Stimulation in Patients with Chronic Local or Widespread Musculoskeletal Pain.
- <sup>2</sup> Murillo et al 2014. Focal vibration in neurorehabilitation. *Eur J Phys Rehabil Med.* 50(2):231-42. Caliendo et al. 2012. Focal muscle vibration in the treatment of upper limb spasticity: a pilot randomized controlled trial in patients with chronic stroke. *Arch Phys Med Rehabil.* 93(9):1656-61. Casale et al. 2014. Localized 100 Hz vibration improves function and reduces upper limb spasticity: a double-blind controlled study. *European Journal of Physical and Rehabilitation Medicine.* *Eur J Phys Rehabil Med.* 50(5):495-504. Noma et al. 2012. Anti-spastic effects of the direct application of vibratory stimuli to the spastic muscles of hemiplegic limbs in post-stroke patients: a proof-of-principle study. *J Rehabil Med.* 44(4):325-30. Mandic et al. 2012. Kinematic analysis of upper-extremity movements after segmental muscle vibration therapy. *Gait & Posture.* 35: S21 - S22. Liepert & Binder 2010. Vibration-induced effects in stroke patients with spastic hemiparesis - a pilot study. *Restor Neurol Neurosci.* 28(6):729-35. Paoloni et al. 2014 Segmental muscle vibration modifies muscle activation during reaching in chronic stroke: A pilot study. *NeuroRehabilitation.* 35(3):405-14. Marconi et al 2011. Long-term effects on cortical excitability and motor recovery induced by repeated muscle vibration in chronic stroke patients. *Neurorehabil Neural Repair.* Jan;25(1):48-60 Tavernese et al 2013. Segmental muscle vibration improves reaching movement in patients with chronic stroke. A randomized controlled trial. *NeuroRehabilitation.* 32(3):591-9.

Celletti & Camerota 2011. Preliminary evidence of focal muscle vibration effects on spasticity due to cerebral palsy in a small sample of Italian children. Clin Ter.162(5):125-8.

<sup>3</sup> Schindler et al. 2002. Neck muscle vibration induces lasting recovery in spatial neglect.

Johannsen et al. 2003. Lasting amelioration of spatial neglect by treatment with neck muscle vibration even without concurrent training.

Kamada et al. 2011. Effects of 5 minutes of neck-muscle vibration immediately before occupational therapy on unilateral spatial neglect.

Karnath et al. 2002. The perception of body orientation after neck-proprioceptive stimulation. Effects of time and of visual cueing.

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