W E L L N E S S

N O V A F O N

Gebrauchsanweisung
User Manual

ES

FR

EN

DE

DEUTSCH

1. VORWORT

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2. PRODUKTBESCHREIBUNG

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2.2. GERÄTETYPEN

2.3. ZUBEHÖR

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2.3.3. Optionales Zubehör

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3.1. WARNUNGEN

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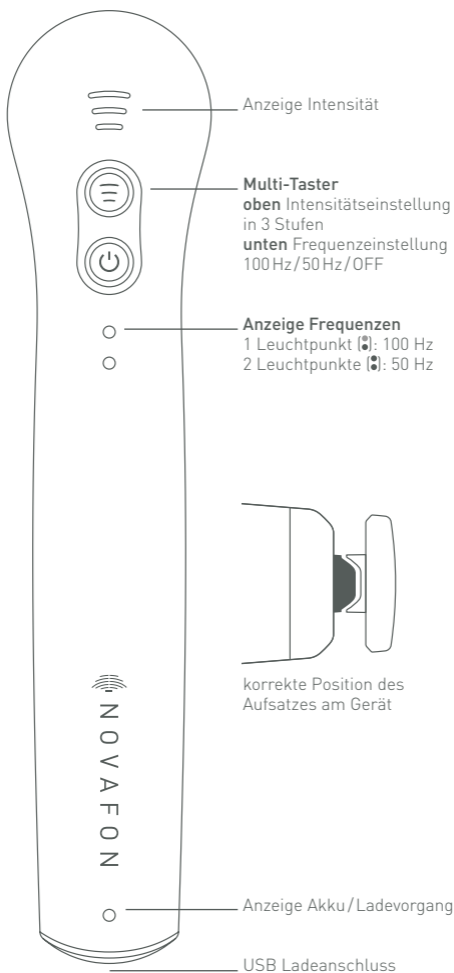
ENGLISH

- 1. PREFACE**
 - 1.1. INTENDED USE
- 2. PRODUCT DESCRIPTION**
 - 2.1. SCOPE OF DELIVERY
 - 2.2. APPLIANCE TYPES
 - 2.3. ACCESSORIES
 - 2.3.1. Disc attachment
 - 2.3.2. Ball attachment
 - 2.3.3. Optional accessories
- 3. IMPORTANT SAFETY INSTRUCTIONS**
 - 3.1. WARNINGS
 - 3.2. OTHER PRECAUTIONS
- 4. CONTRAINDICATIONS OR WHEN NOT TO USE THE APPLIANCE**
- 5. OPERATION**
 - 5.1. FIRST USE
 - 5.2. ADJUSTING THE INTENSITY
 - 5.3. CHARGING
 - 5.4. ATTACHMENTS
- 6. APPLICATION & USE**
- 7. TECHNICAL SPECIFICATIONS**
 - 7.1. SYMBOLS USED
 - 7.2. TECHNICAL SPECIFICATIONS
- 8. SERVICE & MAINTENANCE**
 - 8.1. CLEANING & STORAGE
 - 8.2. TROUBLESHOOTING
 - 8.3. SAFETY CHECK AND MAINTENANCE
 - 8.4. DISPOSAL
- 9. WARRANTY**
- 10. CUSTOMER SERVICE**

1. PREFACE

Congratulations on your purchase of a NOVAFON vibration sound wave appliance. Our appliances have delivered excellent performance and reliability for decades. Please read the following instructions carefully to ensure that you can get the most out of your NOVAFON vibration sound wave appliance and enjoy using it for a long time.

NOTE: Before operating your NOVAFON appliance please read this user manual completely to familiarize yourself with operating precautions in order to make the best and safe use of your appliance.

SAVE THESE INSTRUCTIONS FOR LATER REFERENCE.

1.1. INTENDED USE

NOVAFON appliances are meant to create wellbeing. The combined physiological effect of skin contact together with the gentle stimulation and the warm feeling of the vibration massage can, for many

users, relaxe the muscles in the contact area and temporary relief minor aches and pains; thus, creating a feeling of wellbeing.

2. PRODUCT DESCRIPTION

2.1. SCOPE OF DELIVERY

The scope of delivery includes a battery powered NOVAFON vibration sound wave appliance (see section 2.2), a disc attachment, a ball attachment, a power plug, a USB-C charging cable and instructions for use.

2.2. APPLIANCE TYPES

NOVAFON appliances have been used for decades for smoothing pains and other aches. The application of the vibration massage can be relaxing or stimulating, adding to the wellness feeling.

A stimulating effect is generated by transmitting gentle vibrations to skin surfaces, muscles, cartilage and tissues, which can often be felt for several minutes up to hours after the application.

The application is very easy. The appliances allow you to choose between a more continuous massage (100 Hz) and more of a tapping massage (50 Hz). The three intensity levels (light, medium, strong) allow the treatment to be individually adapted.

The different appliance types differ in their color and the intensity of the vibrations (see table below):

	DAS NOVAFON	DAS NOVAFON power
Multi-function button	ON/OFF, (●) and (☰)	ON/OFF, (●) and (☰)
Frequency setting	Level 1 (●), 100 Hz Level 2 (●), 50 Hz	Level 1 (●), 100 Hz Level 2 (●), 50 Hz
Intensity setting	light (☰), medium (☰) strong (☰)	light (☰), medium (☰) strong (☰)
Intensity	●●●●●	●●●●●
Color	white	anthracite gray
Lithium-ion battery	Capacity 2400 mAh Voltage 3,7 V	Capacity 3400 mAh Voltage 3,7 V

2.3. ACCESSORIES

NOVAFON appliance are equipped with two interchangeable attachments as standard. NOVAFON GmbH also offers other attachments that are not included in this set. Please contact your dealer or visit www.novafon.com for more information. NOVAFON appliances may only be used with original NOVAFON accessories.

THE USE OF ACCESSORIES FROM OTHER MANUFACTURERS

CAN IMPAIR THE FUNCTION AND PERFORMANCE OF NOVAFON APPLIANCES.

2.3.1. Disc attachment

The disc attachment is used in the treatment of larger body parts that require broader strokes. It is included in the delivery and also available as a spare part.

2.3.2. Ball attachment

The ball attachment can be used for smaller areas and targeted treatment. It is included in the delivery and also available as a spare part.

2.3.3. Optional accessories

On our website you will find additional attachments for various treatment goals. There is also a charging stand and USB cover caps for your battery powered NOVAFON appliance. Note that not all attachments may be available in your country.

NOTE: Depending on your country the appliance might be additionally equipped with a second power plug with country specific connectors. These are available as spare parts, too.

3. IMPORTANT SAFETY INSTRUCTIONS

Before starting to use the appliance, please read the user manual and these safety instructions carefully and strictly follow them.

3.1. WARNINGS

When using an electrical appliance basic precautions should always be followed including the following to reduce the risk of burns, fires, electric shock, possibly severe injury and possible death to the user or other people:

⚠ WARNING: Do not use the NOVAFON appliance during charging.

⚠ WARNING: Always store the appliance in a dry and clean location.

⚠ WARNING: Before connecting the power plug to the mains power source, check that the mains voltage corresponds to the voltage specification on the power plug. Connect the NOVAFON appliance only to a power outlet that delivers the correct voltage.

⚠ WARNING: To charge the appliance, use only the charging cable and power supply included in the delivery or the separately available charging stand. Use the correct connector plug, if applicable.

⚠ WARNING: Always unplug the appliance from the electrical outlet after charging and before cleaning.

⚠ WARNING: Ensure that devices are completely dry and with no cleaning agent, disinfectant or oil/cream residues before using the charging stand or charging cable.

⚠ WARNING: Do not bring the appliance and charging accessories (plug, charging cable, charging stand) into contact with hot surfaces (e.g. heater, hotplate, etc.). The housing of the appliance or adapter or the insulation of the cable could be damaged.

⚠ WARNING: Do not use the appliance when taking a shower or bath.

⚠ WARNING: Do not operate where aerosol (spray) products are

being used, in other explosive atmospheres or where oxygen is being administered.

⚠ WARNING: Do not place or drop the appliance into water or other liquids.

⚠ WARNING: Do not reach for a appliance that has fallen into water, especially when the charging cable is connected to a power outlet. Unplug immediately.

⚠ WARNING: Do not place or store appliance where it can fall or be pulled into a tub, pool or sink.

⚠ WARNING: Protect the appliance, charger and charging stand from drop impact and shocks.

⚠ WARNING: This appliance may not be modified without the manufacturer's permission.

⚠ WARNING: NOVAFON appliances may only be used with original NOVAFON accessories. The use of accessories from other manufacturers can impair the function and performance of NOVAFON appliances. This also applies to the power adapter and charging cables.

⚠ WARNING: Do not use the appliance and either discard or return to your dealer for examination if any of the following conditions occur:

- if the appliance fails to work correctly
- if the appliance is damaged, e.g. after having been dropped
- if the appliance was immersed in water or other liquids
- if the appliance exhibits other safety-related problems, such as emitting smoke, unusual or excessive heat etc.

⚠ WARNING: Do not use the power plug, charging cable or charging stand:

- in the event of visible damage or after dropping the power plug, charging cable or charging stand
- if the power plug, charging cable or charging stand has been exposed to extreme moisture
- if liquid has penetrated the unit

⚠ WARNING: Never charge the appliance for more than 24 hours at a time. Do not leave the appliance unattended. Disconnect the appliance from the power supply at night. This also applies when using the charging stand.

⚠ WARNING: Do not place the appliance on flammable surfaces when charging.

⚠ WARNING: Never leave the appliance unattended when it is turned on.

⚠ WARNING: Do not clamp or pinch any part of your body, especially your fingers, between the attachment and the appliance. During the treatment, hold the appliance only by the appliance handle.

⚠ WARNING: Never drop or insert any foreign object into any opening on this appliance. Do not insert any objects (including wires) into openings in the appliance.

⚠ WARNING: Wireless communication devices (such as Wi-Fi stations, mobile phones, cordless phones, and related base stations) can affect the appliance and should be kept at least 0.5 m (2 feet) away from the appliance during use. This also applies to cables and plug-in power supplies.

⚠ WARNING: Careful supervision is necessary when used on or by

children, immobile or disabled individuals. NOVAFON appliances should only be used by users from the age of 18 years who have read the instructions and are competent to use the appliance safely.

⚠ WARNING: Do not use on sleeping or unconscious individuals.

⚠ WARNING: Keep the appliance out of reach of children when not in use.

⚠ WARNING: Small parts (e.g. cover caps) could come loose and be swallowed. Risk of suffocation!

⚠ WARNING: Keep children away from packaging materials. There is a risk of suffocation.

⚠ WARNING: Do not use in the genital area, this appliance is not intended as an intimate pleasure toy.

⚠ WARNING: Do not use outdoors or in wet or damp areas.

⚠ WARNING: Never operate the power plug or charging stand in environments with high humidity or in the presence of moisture. Please note, that moisture is, for example, also generated by nebulisers.

⚠ WARNING: Do not operate under blankets, pillows, towels, clothing etc. or anyplace where ventilation is limited as the appliance could overheat and thus cause fire, electric shock or other injuries to people.

⚠ WARNING: Use the appliance only for its intended use as described in this manual.

⚠ WARNING: Avoid contact with mucous surfaces or the eyes.

3.2. OTHER PRECAUTIONS

NOTE: Use the initial session to figure how the body responds to vibration therapy. If you experience any discomfort, dizziness or nausea stop session immediately.

NOTE: Stop using the NOVAFON appliance immediately when you start feeling unwell, nauseous or dizzy and consult your physician.

NOTE: Do not use the appliance if any of the contraindications described in chapter 4 apply to you.

NOTE: Do not use the appliance for more than 20 minutes per application. Leave it to cool down for at least 15 minutes before restarting.

NOTE: Make sure to secure the attachments fully onto the NOVAFON appliance and keep it on during treatment.

NOTE: Make sure that no substances, such as liquids, creams or the like, can penetrate into the appliance or the power plug.

NOTE: To prevent damage, make sure that the appliance and charging stand are standing or resting on a stable surface during the charging process.

NOTE: Do not twist the appliance in the charging stand. This may cause damage.

NOTE: Protect the appliance against dust, lint and other particles. Penetration of dust, lint or other particles may cause the appliance to fail.

NOTE: To prevent premature wear, only store the appliance in a dry and clean place that is protected against UV light (incl. sunlight).

NOTE: Use only non-alcoholic cleaning supplies. Alcohol can attack the surfaces of the housing and the attachments.

4. CONTRAINDICATIONS OR WHEN NOT TO USE THE APPLIANCE

The NOVAFON vibration sound wave appliance is intended for the application on individual body parts. Do not use the appliance if one or more of the following apply to you:

- Open wounds/eczema (in the body region to be treated)
- Arteriosclerosis (in the body region being treated)
- Current episodes of epilepsy
- Pregnancy
- Implants (in the body region being treated)
- Heart pacemakers
- Thromboses (in the body region being treated)
- Brain pacemakers
- Cardiac arrhythmia
- Tumours, cancerous and not cancerous
- Acute inflammations (in the body region being treated)
- Acute episodes of inflammatory diseases

SIDE EFFECTS:

Please note that in rare cases the following side effects may occur:

- Increased pain/unpleasant sensation
- Nausea/dizziness
- Skin irritation
- Reflex/involuntary muscle contractions

If you are unsure or have any doubts whether the NOVAFON vibration sound wave appliance is suitable for you, consult your physician or treating therapist.

If you suffer from diabetes consult your physician before use.

Immediately stop session if you start feeling nauseous, dizzy or otherwise unwell or if pain in the treated area increases. Do not restart sessions without your physician's approval.

If your skin is reddening under the application and goes away within 30 minutes of the session ending, this is no reason to worry, as the vibration is stimulating blood circulation which is causing the redness. If the reddening of the skin stays after 30 minutes or is painful, consult your physician.

DO NOT USE for treating pains or swellings of unknown origin. In that case consult a medical professional for clarification and diagnosis.

5. OPERATION

5.1. FIRST USE

Charge the appliance fully before using it for the first time (section 5.3). Switch on the NOVAFON appliance by pressing the on/off button once. The appliance starts at 100 Hz (🔊) by default. Pressing the button again decreases the frequency to 50 Hz (🔊). The current frequency selection is indicated by the LED display (frequencies). Press the button a third time to switch off the appliance.

5.2. ADJUSTING THE INTENSITY

Adjust the intensity by pressing the Intensity Setting Button repeatedly. You can choose between three intensity levels: light (🔊),

medium (🔊) and strong (🔊). The intensity level is indicated by the LED display (intensity). The appliance always starts at the lowest intensity level (🔊). This also corresponds to the NOVAFON treatment recommendation, i.e. starting with a low intensity and gradually increasing it according to need and personal comfort.

NOTE: The best vibration is achieved when the appliance is operated without additional contact pressure. If your appliance switches to a lower level by itself – or you're unable to increase the intensity – this is probably because your battery level is low. If this is the case, please recharge your appliance.

5.3. CHARGING

To charge the appliance, only use the included power adapter and USB-C cable or the NOVAFON Charging Stand (not included). To charge the appliance, first connect the cable to the appliance or to the charging stand and then to the power adapter suitable for your power connection. Only then plug the mains adapter into the socket. Charge the appliance until the charge display (LED) no longer flashes. Do not use the NOVAFON appliance during charging. As soon as the LED display no longer flashes, the NOVAFON appliance is fully ready to use. The appliance has a low battery display. If the charge display (LED) starts flashing rapidly, you can continue using your NOVAFON appliance for another 10 minutes. The appliance then switches off automatically. If you use the charging stand for charging, make sure to place the appliance carefully on the USB port. When removing, hold the charging stand down and pull the appliance out carefully. The appliance is charging when the display (LED) lights up. Avoid twisting the appliance in the charging stand as this will prevent damage.

5.4. ATTACHMENTS

The attachments are fixed to the appliance via a plug connection. They can easily be detached by turning them a quarter turn and pulling them gently off the appliance. The attachments click simply onto the appliance. The attachments are in the correct position when the brackets on the appliance and the attachment lie seamlessly on top of each other. The integrated magnets help you to find the final position. Before starting treatment, make sure that the respective attachment is engaged in the intended final position.

Please refer to the drawing on page 26 for the correct position of the attachment on the appliance.

6. APPLICATION & USE

The NOVAFON appliance can be used directly on the skin or through fabric/clothing. Guide the NOVAFON appliance slowly over the affected area in circular or sweeping motion. You can also use the appliance selectively and rest on the spot (for up to 60 seconds or until discomfort is felt) to achieve a feeling of relieve from pain and aches. Please note that you should only guide the appliance with its own weight over the affected area and should not apply additional pressure as this will reduce the vibration effect. Do not use the appliance for more than 20 minutes at a time, and switch it off for at

least 15 minutes to prevent overheating. When applied to sensitive areas (e.g. face, neck, head) or several body parts at one session, the application duration should be reduced and adjusted to accommodate the individual needs. The NOVAFON appliance can be used on all body parts. Avoid contact with the temple areas or the eyes. Do not use on genitals.

The session should feel pleasant and relaxing at all times. Stop the application or change the position of the NOVAFON appliance if you feel the application is painful or unpleasant.

DO NOT USE THE NOVAFON APPLIANCE IF ANY OF THE ABOVE CONTRAINDICATIONS APPLY TO YOU.

Use of liniments: If you want to use any liniments, only use creams, ointments or cosmetics indicated for massage application. Apply only in a thin layer to the skin or place a thin cloth between the NOVAFON appliance and the skin. To prevent damage, ensure that fluids or residues cannot seep into the NOVAFON appliance. Do not use liniments that contain aromatic oils or alcohol as these can cause deterioration of the housing, the attachments and the charging equipment thus damaging the appliance.

It is normal that the attachments and even the appliances head become warm to the touch during use. This warmth is caused by the mechanics in the appliance. If the head becomes hot, turn the appliance off and let it cool down for at least 15 min before using it again, if the appliance continues to be hot, immediately stop using and either discard or return to your dealer.

7. TECHNICAL SPECIFICATIONS

7.1. SYMBOLS USED

~ Alternating current (AC)

≡ Direct current (DC)

☐ Appliance with protection class II according to DIN EN 61140

Hz Hertz

KB Short-term operation

mA Milliampere-hour

A Amp

V Volt

♻ Local laws might require you to dispose of this appliance at specialized facilities or collection points only. Please follow your local laws on disposal of electronic appliances.

SN Displays the serial number of the manufacturer so that a specific appliance can be identified

🔥 Temperature limit: Refers to the temperature limits to which the appliance can be safely exposed.

📖 Refer to the instructions

🏠 Use inside only

⚠ General warning symbols

📶 Intensity setting in 3 stages

• Frequency setting, 100 Hz/50 Hz/OFF

7.2. TECHNICAL SPECIFICATIONS

- Operating conditions: 5 °C (41 °F) to 30 °C (86 °F); 15 % to 93 % relative humidity, non-condensing; 700 hPa to 1060 hPa air pressure
- Storage/transport: -5 °C (23 °F) to 40 °C (104 °F); up to 93 % relative humidity, non-condensing; 500 hPa to 1060 hPa air pressure
- Operating mode: short time duty 20/15 min
- Operating noise: maximum 70 dBA
- Power plug charger SK3
- Input power: AC 100–240 V, 50–60 Hz, 300 mA
- Output power: DC 5 V, 2 A
- The standard scope of delivery includes a type-F power plug (EU plug). Additional power plugs with other connectors are available as spare parts or are already included in the relevant country sets.
- Charging cable USB-A to USB-C, length: approx. 1 m
- Battery: Lithium-ion battery, service life: 800 charging cycles. With a new battery and average intensity, one battery charge guarantees at least six treatment units of 20 minutes each. Note that frequent use of the highest intensity level may shorten the time the battery will last.
 - DAS NOVAFON battery performance data: Capacity 2400 mAh. Voltage 3.7 V
 - DAS NOVAFON power battery performance data: Capacity 3400 mAh. Voltage 3.7 V
- Conformity with 2015/863/EU, 2014/30/EU and Regulation 2016 No. 1091 (UKCA) is declared.
- IP22 protection class, which states: Protected against access with a finger. Protected against water dripping at an angle (when using the USB cover caps, not included in standard scope of delivery).
- Manufacturer: NOVAFON GmbH, Daimlerstraße 13, 71384 Weinstadt, Germany

8. SERVICE & MAINTENANCE

8.1. CLEANING & STORAGE

WARNING: Do not clean the appliance during charging. Disconnect the appliance from the power supply when cleaning. Remove the attachments from the appliance for cleaning.

Clean the appliance housing and attachments with a clean cloth moistened with drinking quality water or lightly soapy water. You may also use alcohol free wipes. Do not immerse in water or use any harsh cleaners or solvents. Do not use cleaners or wipes that contain alcohol (including ethanol, isopropanol, acetone) of any kind. Do not use paint thinner, benzene or any other lacquer thinner. Leave attachments and the appliance to air dry fully before reattaching.

Do not store your NOVAFON appliance in a moist area. Do not store or place your NOVAFON appliance where it can drop or fall into water.

8.2. TROUBLESHOOTING

Contact our Customer Service if any defects or failures occur (see contact details at the end of this manual).

8.3. SAFETY CHECK AND MAINTENANCE

The NOVAFON device does not require regular inspection or maintenance. To ensure safe operation, however, you should check the device regularly for obvious damage. Please note that there may be national laws that oblige you to carry out safety checks on the device. This does not generally apply to private individuals. If you have any problems with your appliance either discard or return it to your dealer.

8.4. DISPOSAL

This appliance must be disposed of in accordance to Federal and Local State law(s). This appliance must never be disposed of with common household waste. For environmentally sound disposal, the user is obligated to dispose of the old appliance at a designated collection point for the disposal of electrical and electronic equipment. This appliance is subject to EU Directive on Waste Electrical and Electronic Equipment (WEEE).

9. WARRANTY

From the date of delivery, we grant a warranty of 2 years (24 months) for defects resulting from material or production defects. Warranty coverage is subject to the presentation of a proof of purchase. The battery, charging cable, power plug and the attachments is guaranteed for 6 months.

Excluded from the warranty: Damage due to improper use, normal wear and tear and consumption (also valid for accessories).

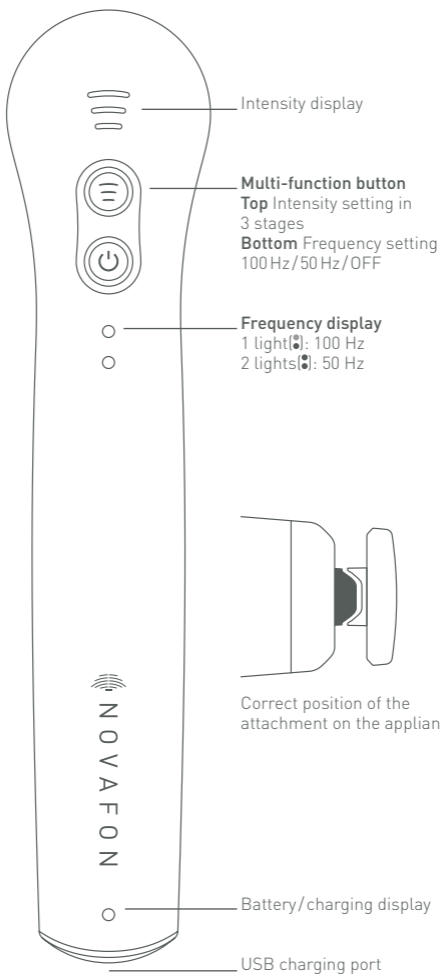
The right of rescission (reversal of the purchase), reduction (discount on the purchase price) or subsequent delivery is initially excluded. However, you have a claim to reduction or conversion at your discretion if we do not remedy a defect that is subject to our warranty within a reasonable period of time. Any damage caused by failure to use the product in accordance with its intended use or resulting from improper or unreasonable use or maintenance is not covered by the warranty. Such damage is the sole responsibility of the user. Failure to comply with this instruction manual will void the warranty. Some states may not allow such limitation, so it may not apply to you. This limited warranty gives you specific and you may have other rights which may vary from state to state.

Get in touch with your dealer or supplier to claim warranty.

10. CUSTOMER SERVICE

NOVAFON appliance require precise adjustment. Consequently, any necessary repairs must be performed exclusively by the NOVAFON Customer Service. Please do not send in your defective appliance without first contacting the dealer by telephone to report the damage and determine the scope of the repair. The repair might be more expensive than a new appliance. If, during the warranty period, the repair of a appliance proves too costly, NOVAFON or its dealer will offer a replacement appliance (to which a 24-months' limited warranty period will apply).

This manual can also be viewed as a PDF file at www.novafon.com.

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- Fabricante: NOVAFON GmbH, Daimlerstrasse 13, 71384 Weinstadt, Alemania