
novafon®

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
Mobility-Set max

Manual for Mobility set max


③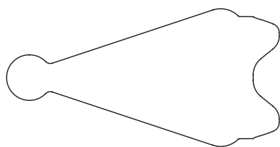

②

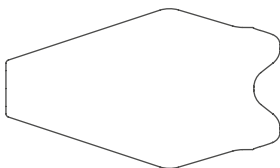①

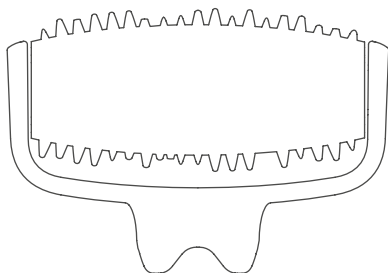Deutsch

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English

1. Preface	8
2. Scope Of Delivery	8
3. Applications & Use	8
3.1 Intended Use	9
3.2 Typical Indications For Use	9
4. Important Instructions	10
5. Material Information	11
6. Durability And Warranty	11
7. Cleaning And Disinfection	11
8. Contraindications / Side Effects	12
9. CE Conformity	13
10. Contact	13

1. Preface

The nova fon ‚Mobility set‘ attachments are designed as an additional extension to the existing attachments included with the nova fon max+ and nova fon max sound wave devices for local vibration therapy. They are intended solely for use in conjunction with such a device. Please read the following pages carefully to ensure proper use.

2. Scope Of Delivery

The scope of delivery includes a Sensory roller max, a pin attachment max, a focus attachment max and instructions for use. The Mobility set is available in black (item no. 7112-007) and white (item no. 7112-040).

3. Applications & Use

The attachments are fixed to the device via a plug connection. The attachments can easily be detached by turn and pulling them gently off the device. The attachments click simply onto the device. The attachments have in the correct position when the brackets on the device and the attachment lie seamlessly on top of each other. The magnets help

you to find the final position. Before starting treatment, make sure that the respective attachment is engaged in the intended final position. Note: The attachments can be used directly on the skin or through fabric/clothing. There is no need for gel or similar.

Note: In sensitive areas of the body, e.g. neck, head, hands, only work at low intensities. Otherwise there may be increased side effects.

3.1 Intended Use

novafon devices and accessories are designed to relieve pains and aid in relaxation of muscles to alleviate symptoms of:

- Temporary and transient pain of the musculoskeletal system
- Chronical pain of the musculoskeletal system
- Neuro Rehabilitation (sensory and motor skills)

3.2 Typical Indications For Use

Note: Vibration therapy primarily relieves symptoms, but does not cure the underlying disease. The product is not intended to replace a doctor's visit.

Note: Local vibration therapy for neurological disorders must only be used by specialised personnel (therapists, doctors, nursing staff) or after thorough instruction by a specialist.

In all cases, local vibration therapy is used as an element in a holistic therapy plan in which muscle relaxation and muscle activation are required for further exercises and training elements. Before using the attachments, please also observe all information from the operating instructions of your novafon sound wave device. **Mind the contraindications** as named in chapter 8 as well as the instructions in chapter 4.

Sensory roller max	Pin attachment max	Focus attachment max
Targeted rolling out of large areas of the body to stimulate surface sensitivity	Precise mobilization, also suitable for cross-frictional applications, e.g. hooking tendons	Focused, selective, in-depth treatment on pain and trigger points
Stimulation of blood circulation in tissue, e.g. scar tissue, and sensory stimulation in case of discomfort	Targeted displacement of tissue, e.g. fasciae, scars, tendons	Effektive Behandlung von Schmerzen in lokal begrenzten Regionen
Treatment of superficial areas of the arms, legs and upper body	Treatment of fine structures on the forearms, hands and feet	Treatment in body regions with pain and trigger points

4. Important Instructions

- Depending on personal sensitivity, the vibration can be perceived with different intensity. Treat particularly sensitive spots at the lowest intensity level, through clothing or a fine cloth.
- Treat a maximum of 20 minutes per unit, then let the novafon sound wave device cool down for 15 minutes.
- In case of discomfort, dizziness, or an increase in the symptoms, stop the treatment.
- Only work at low intensity in sensitive areas of the body, e.g. neck, head and hands. Otherwise you may experience increased side effects.
- Keep attachments out of reach of children.
- Do not use the attachments on mucous membranes and

open wounds.

- Avoid impacts! These can damage the products and hence lead to a risk of injury.
- novafon accessories are only intended for use with novafon vibration sound wave devices. Use with products from other manufacturers is not permitted.
- **Note:** When using these attachments, the noise generated may differ from that of the standard attachments. Do not switch on the device until you have made physical contact.
- Before using the attachments, please also observe all information from the operating instructions of your novafon sound wave device.

5. Material Information

The novafon attachments are made of high-quality plastic suitable for medical applications. Please store in a cool, dry place away from sunlight.

6. Durability And Warranty

The durability and warranty of the attachments will vary depending on the frequency of use. Check the attachments before each application. Replace damaged or deformed products. We recommend replacing the attachments after two years of use. Even much sooner if used frequently. Please note that attachments are considered wear parts and are therefore subject to a limited warranty of 6 months.

7. Cleaning And Disinfection

For professional and clinical use, novafon attachments must be disinfected with a surface disinfectant for medical devices before and after each application. For cleaning, please use alcohol-free disinfectants that are suitable for use on sensitive areas, such as novacleaner from novafon, SinAlc sensitiv from HIMED or RHEOSEPT-WD plus Wipes from

Rheosol. If necessary, wipe it dry with a soft cloth. For the 'pin attachment', we specifically recommend a non-alcoholic disinfectant spray. For the sensory-roller max, we specifically recommend a non-alcoholic disinfectant spray. In case of dirt or if otherwise necessary, clean the attachments with a brush and a neutral cleaner under running water before disinfecting. For domestic, non-clinical use, clean the attachments with a damp cloth, under running water or in a mild soapy solution.

Allow the attachments to dry completely before using them with the appliance again.

8. Contraindications / Side Effects

Treatments with the novafon sound wave devices are not permitted for patients with:

- Open wounds / eczema (in the body region to be treated)
- Arteriosclerosis (in the body region being treated)
- Current episodes of epilepsy
- Pregnancy
- Implants (in the body region being treated)
- Heart pacemakers and defibrillators
- Thromboses (in the body region being treated)
- Brain pacemakers
- Cardiac arrhythmia
- Tumours, cancerous and not cancerous
- Acute inflammations (in the body region being treated)
- Acute episodes of inflammatory diseases

Side effects:

Please note that in rare cases the following side effects may occur:

- Increased pain/unpleasant sensation (occasional)
- Discomfort/ dizziness/ restlessness (occasional)
- Skin irritation (rare)
- Bruising (very rare)
- Reflexes/ involuntary muscle contractions (rare)
- Sore muscles (rare)

The side effects usually disappear after 10 minutes.

Please discontinue the treatment immediately, if you experience any of these side effects. If symptoms persist, please consult a doctor. Please note that pain treatment in particular can lead to an initial worsening of the symptoms before you notice any improvement. Please note that slight reddening and warming is not a cause for concern. This is a desired effect of local vibration therapy, which is caused by the increased blood flow.

9. CE Conformity

The manufacturer declares conformity with the requirements in respect to medical devices of class I according to Regulation 2017/745/EU and UK MDR 2002 in sole responsibility for this product. Please contact us in case you suspect the device to have caused a serious incident. As a user you have the right to report such an incident to your national competent authority (e.g Ministry of health; FDA).

10. Contact

If you have any questions, please contact your local dealer or the manufacturer.

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NOVAFON – Elektromedizinische Geräte GmbH,