N O V A F O N

Gebrauchsanweisung
Sensi – Roller
Manual for Sensory Roller

DEUTSCH

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ENGLISH

SENSORY ROLLER SET - INSTRUCTIONS FOR USE

- | | |
|--------------------------------|------------------------------|
| 1. PREFACE | 6. PRODUCT LIFE |
| 2. SCOPE OF DELIVERY | 7. CLEANING AND DISINFECTION |
| 3. APPLICATIONS & INTENDED USE | 8. CONTRAINDICATIONS |
| 4. IMPORTANT INSTRUCTIONS | 9. CE CONFORMITY |
| 5. MATERIAL INFORMATION | 10. CONTACT |

1. PREFACE

The NOVAFON Sensory Roller set are designed as an extension to the existing attachments supplied with the NOVAFON vibration sound wave devices for local vibration therapy. They are intended solely for use in conjunction with such a device. The Sensory Roller set is available in anthracite gray and white. Please read the following pages carefully to ensure proper use.

2. SCOPE OF DELIVERY

The Sensory Roller set consists of the following items: Adapter, Retaining clip, 3 different coloured rollers, User Manual. The different hardness grades can be identified by the respective colour marking: yellow = soft; orange = medium; red = hard. The set is available in White (Item No. 7017) anthracite gray (7037) and black (7052).

3. APPLICATIONS & USE

Mount the Sensory Roller attachment completely assembled on your NOVAFON sound wave device using the adapter. You can rotate the retaining clip inside the adapter to ensure optimal alignment. Hold the adapter with one hand while turning the attachment with the other hand.

Adjust the orientation so that you can roll straight over the area to be treated while the device sits loosely in your hand. Be careful not to apply any additional pressure as this may interfere with the roller's performance and damage the unit. Adjust the roll speed and pressure intensity as well as the hardness grade of the rollers to the individual requirements of the treatment. Refer to the operating instructions of your NOVAFON sound wave device for the recommended treat-

ment duration.

This is how you change the rollers: If you want to remove the attachment or change the roller, always twist the adapter and its attachment out of the device and only then remove the roller from the fixture. To change the rollers, carefully pull apart the bifurcation of the retaining clip and remove the roller by pulling it slightly upwards on one side. Place the new roller with one side first inside the retaining clip and let the other side snap into place with light pressure from the top. Avoid pinching your fingers between the roller and the retaining clip. While inserting the new roller, make sure that it engages completely. Do not switch on the device until it is resting on the part of the body to be treated, otherwise noise may be generated.

3.1 INTENDED USE

NOVAFON devices and accessories are designed to relieve pains and aid in relaxation of muscles to alleviate symptoms of:

- Temporary and transient pain of the musculoskeletal system
- Chronical pain of the musculoskeletal system
- Neuro Rehabilitation (sensorimotor disorders)
- Voice and swallowing disorders⁴

3.2 TYPICAL APPLICATIONS

Two well-established treatment methods are combined during the Sensory Roller application: The vibrations of the NOVAFON vibration sound wave devices for pain relief as well as the stimulation of the upper skin layers to promote blood circulation through the specific shape of Sensory Roller attachments, which are comparable to a spiky massage ball. The cone-shaped tips of the rollers induce stimuli on the surface of the skin and simultaneously stimulate deeper tissue layers through the vibrations of the sound wave devices. The three different hardness grades of the rollers enable individual adaptation of the stimulus intensity to the treatment goal or the needs of the person receiving the treatment. The different rollers can be exchanged via the retaining clip and attached to the device. The Sensory Rollers are compatible with all NOVAFON sound wave devices with the corresponding adapter. The Sensory Rollers are available in white and anthracite gray. Please read the following pages carefully to ensure proper use. Before using the attachments, please also observe all information from the operating instructions of your NOVAFON sound wave device.

4. IMPORTANT INSTRUCTIONS

- Use the Sensory Roller only in combination with NOVAFON sound wave devices.
- Always follow all the information from the operating instructions of your NOVAFON sound wave device.
- Avoid using creams or gels so as not to block the rolling mechanism.
- Always connect the attachment to the adapter first and then mount it onto your NOVAFON sound wave device.
- If you wish to change or remove the attachment, always disconnect the adapter with the attachment from the device before you detach the attachment from the adapter.
- The attachments can be rotated within the adapter. If you want to rotate your attachment inside the adapter while it is already on your NOVAFON sound wave device, hold the adapter firmly with one hand while you rotate the attachment.
- Be careful not to apply any additional pressure as this may damage the device.
- Be careful when you are changing the rollers, so you do not pinch your fingers.
- Keep the Sensory Roller set dry and protected from light to prevent premature ageing of the material.
- Check the attachments regularly for wear and damage. Damaged attachments must no longer be used. Risk of injury.

5. MATERIAL INFORMATION

The NOVAFON Sensory Roller attachments are made of high-quality plastic specifically designed for medical applications.

6. PRODUCT LIFE

The product life will vary depending on the frequency of use. All parts must be checked for damage and wear before use. Replace damaged or worn products early-on. To avoid premature material ageing, please keep the Sensory Roller set dry and protected from light. Please note that attachments are considered wear parts and are therefore subject to a limited warranty.

7. CLEANING AND DISINFECTION

For professional and clinical use, the NOVAFON Sensory Roller should be sanitised before first use (see chapter on 'Manual Cleaning and Disinfection'). For use on multiple people, the attachments must be sanitised after each application. Please note that machine cleaning of the Sensory Roller is not possible. In domestic, non-clinical use, the attachment may be cleaned with a mild soapy solution as needed or whenever there is visible soiling.

NOTE: If you are using detergents and disinfectants, please observe the relevant usage instructions of the agent of the respective manufacturer. Use only non-alcoholic detergents and disinfectants (VAH or IHO listed) for sensitive surfaces.

MANUAL CLEANING AND DISINFECTION

A. PREPARATION: Remove the Sensory Roller from the retaining clip. Remove adapter.

B. PRETREATMENT AS NEEDED: (e.g. in case of heavy surface contamination)
Required accessories: suitable plastic brush, running water (at least drinking water quality), if needed, neutral cleaning agent or alcohol-free cleaning cloths/wipes (suitable for sensitive surfaces, e.g. Rheosept-WD Plus Wipes, Wachendorff-Chemie).

- Remove surface contamination with a plastic brush under running water. Alternatively, light contamination can be removed with a separate disinfectant wipe.
- Check for cleanliness: If contamination is still visible, repeat the pretreatment using a neutral cleaning agent. Rinse each part of the attachments under running water for neutralisation and then dry them with a lint-free cloth.

C. CLEANING AND DISINFECTION: Make sure that all visible dirt is removed before disinfecting. For cleaning and disinfection of sensitive surfaces, please use a non-alcoholic surface disinfectant (VAH or IHO listed, e.g. SinAlc sensitiv, Hibomed, Rheosept-WD Plus Wipes, Wachendorff-Chemie).

- Thoroughly treat all surfaces of the attachments and the fixture with the disinfectant. Make sure that all surfaces are completely wetted. Follow the manufacturer instructions of the disinfectant you are using.
- If necessary, wipe them dry with a lint-free cloth after the exposure time has elapsed. Allow the attachments to dry completely before using them with the appliance again.

8. CONTRAINDICATIONS

Since the Sensory Rollers must be used as accessories and only in combination with NOVAFON sound wave devices, the contraindications and side effects described in the operating instructions of the NOVAFON sound wave devices apply.

In addition, the following instructions should be observed:

- The Sensory Rollers are not suitable for use on open wounds or mucous membranes.
- Stop treatment if you find it uncomfortable or painful.
- Please note that the appearance of slight reddening and warming is not a cause for concern. This is a desired effect and is caused by the increased blood flow.

9. CE CONFORMITY

The manufacturer declares conformity with the requirements in respect to medical devices of class I according to Regulation 2017/745/EU and UK MDR 2002 in sole responsibility for this product. Please contact us in case you suspect the device to have caused a serious incident. As a user you have the right to report such an incident to your national competent authority (e.g Ministry of health; FDA).

10. CONTACT

If you have any questions, please contact your local dealer or the manufacturer.

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NOVAFON Elektromedizinische Geräte GmbH,