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### EN USER MANUAL

#### ORIGINAL NOVAFON INTRAORAL ATTACHMENTS

##### 1. PREFACE

##### 2. PRODUCT DESCRIPTION

##### 3. APPLICATIONS & USE

##### 4. IMPORTANT INSTRUCTIONS

##### 5. CONTRAINDICATIONS AND SIDE EFFECTS

##### 6. CLEANING AND DISINFECTION

##### 7. CONTACT

#### 6. CLEANING AND DISINFECTION

##### PROCEDURE

- Cleaning and disinfection: manual
- Cleaning and disinfection: by machine

**Warning information** The attachments must be cleaned and, if necessary, disinfected before the first application. For use on several people, the products must be cleaned and disinfected after each application.

**Restriction regarding reprocessing** No special requirements.

##### INSTRUCTIONS

**Application location** Application in a closed container.

**Storage and transport** It is highly recommended to transport contaminated intraoral attachments in a closed container. It is recommended to carry out the reprocessing of intraoral attachments as soon as possible, within a maximum of 2 hours after use.

**Preparation** Wear personal protective equipment (gloves, water repellent lab coat, face mask, or safety glasses). Disassemble all detachable intraoral attachments into their individual parts.

**Pretreatment** Equipment: Plastic brush (e.g., Interlock, #09098 and Interlock, #09050), running water (at least drinking water quality)

Procedure: Brush coarse surface contamination on the attachments and the adapter (thread and cavity) under running water.

##### MANUAL

**Manual cleaning** Equipment: Cleaning agent (multistage enzymatic cleaner, e.g. ECOLAB Sekusept® MultiEnzyme P [ecolab.com]), plastic brush (e.g. Interlock, #09098 and Interlock, #09050), running water (at least drinking water quality), tray for cleaning agent.

1. Apply the cleaning solution according to the manufacturer's instructions (e.g. 2% Sekusept® MultiEnzyme P).

2. Submerge all parts of the attachment completely.

3. Remove surface contamination with the plastic brush, by scrubbing all surfaces of the attachment and the adapter.

4. Treatment time in the cleaning solution according to the manufacturer's instructions of the cleaning agent (e.g. 2% Sekusept® MultiEnzyme P, at least 15 min).

5. Remove all parts from the cleaning solution.

6. Rinse each part of the attachments under running water for 30 sec. Check for cleanliness and, if there is still visible contamination, repeat the pretreatment and the aforementioned manual steps.

##### Manual disinfection

**Equipment:** Virucidal instrument disinfectant (VAH or IHO listed) based on peroxide compounds or quaternary compounds and alkylamine/alkylamin derivatives (e.g. ECOLAB Sekumatic® or Dr. Schumacher, Perfekta active), disinfectant tray

1. Apply the disinfectant solution according to the disinfectant manufacturer's instructions (e.g. Sekusept® aktiv, 2% / 2% Perfekta active).

2. Submerge all parts of the attachments completely.

3. Treatment time in the disinfectant solution according to the manufacturer's instructions for the disinfectant (e.g. at least 5 minutes for Sekusept® aktiv and at least 30 minutes for Perfekta active).

4. Remove all parts of the attachment from the disinfectant solution.

5. Rinse each item for 1 minute under running water. Wipe all parts with a lint-free disposable cloth for drying and, if necessary, dry it with medical compressed air

**BY MACHINE**

**Cleaning and disinfection:** by machine

**Note:** Coarse surface contamination on the attachments must be removed before preparation by machine (see pretreatment: manual). Cleaning and disinfecting device according to DIN EN ISO 15883-1+2 with thermal program (temperature 90 °C to 95 °C).

- Mildly alkaline cleaner (e.g. ECOLAB Sekumatic® MultiClean)

1. Place all the individual parts of the attachments in a suitable strainer bowl or on the load carrier in such a way that all inner and outer surfaces are cleaned and disinfected.

2. Close the RDG and start the programme; for programme sequence see table below.

3. At the end of the programme, remove the products.

4. Check the cleanliness and dryness of the load (all parts) and, if necessary, dry with medical compressed air. For residual contamination, carry out the manual pre-treatment and manual cleaning.

**Programme sequence**

Water Dosage Time Temperature

**Prewash**

cold water 5 min.

**Cleaner dosage**

According to manufacturer

**Cleaning**

completely desalinated water 10 min. 55 °C

**Wash**

completely desalinated water 2 min.

**Disinfecting**

completely desalinated water A0 value > 3,000<sup>1</sup> (e.g. 90 °C, 5 min.)

**Drying**

15 min. Up to 120 °C

<sup>1</sup> Authorities may adopt other implementing provisions (parameters for disinfection performance) within their jurisdiction.

The material manufacturer has confirmed the resistance to steam sterilization at 121 °C and 134 °C.

**Maintenance, inspection, and testing**

Check all parts of the attachments for damage and wear. Damaged medical devices may no longer be used and must be sorted out.

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