# NOVAFON

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# **ENGLISCH**

## **MANUAL FOR 'REHABILITATION ATTACHMENTS'**

- |                                  |                                     |
|----------------------------------|-------------------------------------|
| <b>1. PREFACE</b>                | <b>6. IMPORTANT INSTRUCTIONS</b>    |
| <b>2. SCOPE OF DELIVERY</b>      | <b>7. CLEANING AND DISINFECTION</b> |
| <b>3. MATERIAL INFORMATION</b>   | <b>8. CONTRAINDICATIONS</b>         |
| <b>4. SHELF LIFE</b>             | <b>9. CE CONFORMITY</b>             |
| <b>5. APPLICATIONS &amp; USE</b> | <b>10. CONTACT</b>                  |

### **1. PREFACE**

The attachments of the NOVAFON 'Rehabilitation Set' (cone attachment, pin attachment, disk attachment XL - item no. 7016) are designed as an extension to the existing attachments supplied with the NOVAFON sound wave devices for local vibration therapy. They are intended solely for use in conjunction with such a device. Please read the following pages carefully to ensure proper use.

### **2. SCOPE OF DELIVERY**

The scope of delivery includes a cone attachment (item no. 7014), a pin attachment (item no. 7013), a disc attachment XL (item no. 7015) and instructions for use.

### **3. MATERIAL INFORMATION**

The NOVAFON rehabilitation attachments are made of high-quality plastic specifically designed for medical applications. Please store in a cool, dry place away from sunlight.

### **4. SHELF LIFE**

The shelf life of the attachments will vary depending on the frequency of use. Check the attachments before each application. Replace damaged or deformed products. We recommend replacing the attachments after two years of use. Even much sooner if used frequently.

### **5. APPLICATIONS & USE**

The attachments are fixed to the device via a plug connection. The attachments can easily be detached by turning them a quarter turn and pulling them gently off the device. The attachments click simply onto the device. The attachments have in the correct position when the brackets on the device and the attachment lie seamlessly on top of each other. The magnets help you to find the final position. Before starting treatment, make sure that the respective attachment is engaged in the intended final position. Prior to first use, please pay attention to the contraindications listed in Chapter 8 as well as the information in Chapter 6. Note that vibration therapy primarily alleviates symptoms but does not cure the underlying condition. The use of REHAB attachments is no substitute for medical consultation.

Note: The attachments can be used directly on the skin or through fabric/clothing. There is no need for gel or similar.

## Application examples:

Cone attachment	Pin attachment	Disk attachment XL
Targeted treatment for trigger/pain/acupressure points	Special sensory input for desensitisation/sensation disturbance	Large area treatment of the back and thighs
Precise movement of tissue layers and scar tissue	Circular movements to promote circulation	Sensory training from head to toe
Treatment of fine structures on the face, hands and feet	Treatment of pain on the head/hairline via brush-like surface	Full body treatment after sports or large-area pain

## 6. IMPORTANT INSTRUCTIONS

- Depending on personal sensitivity, the vibration can be perceived with different intensity. Treat particularly sensitive spots at the lowest intensity level, through clothing or a fine cloth.
- Treat a maximum of 20 minutes per unit, then let the NOVAFON sound wave device cool down for 15 minutes.
- In case of discomfort, dizziness, or an increase in the symptoms, stop the treatment.
- Keep attachments out of reach of children.
- Do not use the attachments on mucous membranes and open wounds.
- Avoid impacts! These can damage the products and hence lead to a risk of injury.
- Before using the attachments, please also observe all information from the operating instructions of your NOVAFON sound wave device.

## 7. CLEANING AND DISINFECTION

For professional and clinical use, NOVAFON REHA attachments must be disinfected with a surface disinfectant for medical devices before and after each application. As a general rule, NOVAFON recommends alcohol-free disinfectants that are suitable for use on sensitive surfaces, such as SinAlc sensitive by HIMED or RHEOSEPT-WD plus wipes by Rheosol. If necessary, wipe it dry with a soft cloth. For the 'pin attachment', we specifically recommend a non-alcoholic disinfectant spray, e.g. SinAlc Sensitive by HIMED or a comparable product for surface disinfection of medical devices. In case of dirt or if otherwise necessary, clean the attachments with a brush and a neutral cleaner under running water before disinfecting. For domestic, non-clinical use, clean the attachments with a damp cloth, under running water or in a mild soapy solution.

## 8. CONTRAINDICATIONS / SIDE EFFECTS

Treatments with the NOVAFON sound wave devices are not permitted for patients with:

- Open wounds / eczema (in the body region to be treated)
- Arteriosclerosis (in the body region being treated)
- Current episodes of epilepsy
- Pregnancy
- Implants (in the body region being treated)
- Heart pacemakers
- Thromboses (in the body region being treated)
- Brain pacemakers
- Cardiac arrhythmia
- Tumours
- Acute inflammations (in the body region being treated)
- Acute episodes of inflammatory diseases

Side effects:

Please note that in rare cases the following side effects may occur:

- Increased pain / unpleasant sensation
- Nausea / dizziness
- Skin irritation
- Reflex / involuntary muscle contractions

Please discontinue the treatment immediately, if you experience any of these side effects. If symptoms persist, please consult a doctor. Please note that pain treatment in particular can lead to an initial worsening of the symptoms before you notice any improvement. Please note that slight reddening and warming is not a cause for concern. This is a desired effect of local vibration therapy, which is caused by the increased blood flow.

## 9. CE CONFORMITY

The manufacturer declares conformity with the requirements in respect to medical devices of class I according to Regulation 2017/745/EU in sole responsibility for this product.

## 10. CONTACT

If you have any questions, please contact your local dealer or the manufacturer.

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