Gebrauchsanweisung
Sensi-Roller
Manual for Sensory Roller

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- | | |
|---------------------------|----------------------------|
| 1. PREFACE | 5. CONTRAINDICATIONS |
| 2. PRODUCT DESCRIPTION | 6. HYGIENE |
| 3. APPLICATION | 7. MATERIAL AND SHELF LIFE |
| 4. IMPORTANT INSTRUCTIONS | 8. CE CONFORMITY |

1. PREFACE In combination with the NOVAFON sound wave devices, the NOVAFON Sensory Rollers are designed to stimulate surface and deep-tissue sensitivity. Two well-established treatment methods are combined during the Sensory Roller application: The vibrations of the NOVAFON sound wave devices for pain relief as well as the stimulation of the upper skin layers to promote blood circulation through the specific shape of Sensory Roller attachments, which are comparable to a spiky massage ball. Please read the following pages carefully to ensure proper use. Before using the attachments, please also observe all information from the operating instructions of your NOVAFON sound wave device.

2. PRODUCT DESCRIPTION The cone-shaped tips of the rollers induce stimuli on the surface of the skin and simultaneously stimulate deeper tissue layers through the vibrations of the sound wave devices. The three different hardness grades of the rollers enable individual adaptation of the stimulus intensity to the treatment goal or the needs of the person receiving the treatment.

The Sensory Roller set consists of the following items:

Adapter, Retaining clip, 3 different coloured rollers, User Manual

The different hardness grades can be identified by the respective colour marking: yellow = soft; orange = medium; red = hard

The different rollers can be exchanged via the retaining clip and attached to the device. The Sensory Rollers are compatible with all NOVAFON sound wave devices with the corresponding adapter.

3. APPLICATION Mount the Sensory Roller attachment completely assembled over the adapter on your NOVAFON sound wave device. You can rotate the retaining clip inside the adapter to ensure optimal alignment. Hold the adapter with one hand while turning the attachment with the other hand. Adjust the orientation so that you can roll straight over the area to be treated while the device sits loosely in your hand. Be careful not to apply any

additional pressure as this may interfere with the roller's performance and damage the unit. Adjust the roll speed and pressure intensity as well as the hardness grade of the rollers to the individual requirements of the treatment. Refer to the operating instructions of your NOVAFON sound wave device for the recommended treatment duration.

This is how you change the rollers: If you want to remove the attachment or change the roller, always twist the adapter and its attachment out of the device and only then remove the roller from the fixture.

To change the rollers, carefully pull apart the bifurcation of the retaining clip and remove the roller by pulling it slightly upwards on one side. Place the new roller with one side first inside the retaining clip and let the other side snap into place with light pressure from the top. Avoid pinching your fingers between the roller and the retaining clip. While inserting the new roller, make sure that it engages completely.

4. IMPORTANT INSTRUCTIONS

- Use the Sensory Roller only in combination with NOVAFON sound wave devices.
- Always follow all the information from the operating instructions of your NOVAFON sound wave device.
- Avoid using creams or gels so as not to block the rolling mechanism.
- Always connect the attachment to the adapter first and then screw it onto your NOVAFON sound wave device.
- If you wish to change or remove the attachment, always unscrew the adapter with the attachment from the device before you detach the attachment from the adapter. Otherwise, the device may be damaged.
- The attachments can be rotated within the adapter. If you want to rotate your attachment inside the adapter while it is already on your NOVAFON sound wave device, hold the adapter firmly with one hand while you rotate the attachment. Otherwise, the device may be damaged.
- Be careful not to apply any additional pressure as this may damage the device.
- Be careful when you are changing the rollers, so you do not pinch your fingers.
- Keep the Sensory Roller set dry and protected from light to prevent premature ageing of the material.

5. CONTRAINDICATIONS Since the Sensory Rollers must be used as accessories and only in combination with NOVAFON sound wave devices, the contraindications and side effects described in the operating instructions of the

NOVAFON sound wave devices apply.

In addition, the following instructions should be observed:

- The Sensory Rollers are not suitable for use on open wounds or mucous membranes.
- Stop treatment if you find it uncomfortable or painful.
- Please note that the appearance of slight reddening and warming is not a cause for concern. This is a desired effect and is caused by the increased blood flow.

6. HYGIENE For professional and clinical use, the NOVAFON Sensory Roller should be sanitised before first use (see chapter on 'Manual Cleaning and Disinfection'). For use on multiple people, the attachments must be sanitised after each application. Please note that unfortunately machine cleaning of the Sensory Roller is not possible.

In domestic, non-clinical use, the attachment may be cleaned with a mild soapy solution as needed or whenever there is visible soiling.

NOTE: If you are using detergents and disinfectants, please observe the relevant usage instructions of the agent of the respective manufacturer. Use only non-alcoholic detergents and disinfectants (VAH or IHO listed) for sensitive surfaces.

MANUAL CLEANING AND DISINFECTION

A. PREPARATION: Remove the Sensory Roller from the retaining clip. Remove adapter.

B. PRETREATMENT AS NEEDED: (e.g. in case of heavy surface contamination) Required accessories: suitable plastic brush, running water (at least drinking water quality), if needed, neutral cleaning agent or alcohol-free cleaning cloths/wipes (suitable for sensitive surfaces, e.g. Rheosept-WD Plus Wipes, Wachendorff-Chemie).

- Remove surface contamination with a plastic brush under running water. Alternatively, light contamination can be removed with a separate disinfectant wipe.
- Check for cleanliness: If contamination is still visible, repeat the pretreatment using a neutral cleaning agent. Rinse each part of the attachments under running water for neutralisation and then dry them with a lint-free cloth.

C. CLEANING AND DISINFECTION: Make sure that all visible dirt is removed before disinfecting. For cleaning and disinfection of sensitive surfaces, please use a non-alcoholic surface disinfectant (VAH or IHO listed, e.g.

SinAlc sensitiv, Hibomed, Rheosept-WD Plus Wipes, Wachendorff-Chemie)

- Thoroughly treat all surfaces of the attachments and the fixture with the disinfectant. Make sure that all surfaces are completely wetted. Follow the manufacturer instructions of the disinfectant you are using.
- If necessary, wipe them dry with a lint-free cloth after the exposure time has elapsed.

7. MATERIAL AND SHELF LIFE The NOVAFON Sensory Roller attachments are made of high-quality plastic specifically designed for medical applications. The shelf life will vary depending on the frequency of use. All parts must be checked for damage and wear before use. Replace damaged or worn products early-on. To avoid premature material ageing, please keep the Sensory Roller dry and protected from light.

8. CE CONFORMITY The product meets the requirements for Class I medical devices in accordance with Directive 93/42/EEC. The manufacturer declares conformity under sole responsibility in accordance with Annex VII of the Directive.

9. CONTACT If you have any questions, please contact your local dealer or the manufacturer.

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Novafon GmbH