Supported by:

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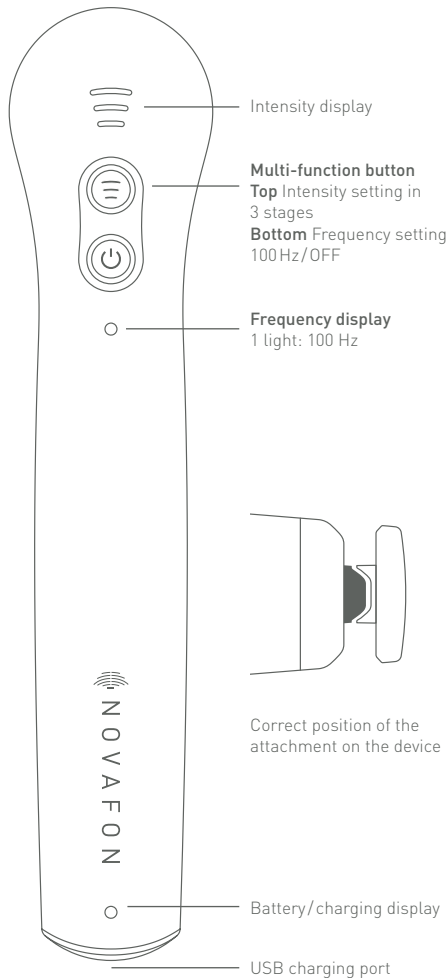
## 1. PREFACE

## 2. PRODUCT DESCRIPTION

- 2.1. SCOPE OF DELIVERY
- 2.2. DEVICE TYPES
- 2.3. ACCESSORIES
  - 2.3.1. Disc attachment
  - 2.3.2. Ball attachment
  - 2.3.3. Extension handle
  - 2.3.4. Optional accessories
  - 2.3.5. Combination with other products

## 3. OPERATION

- 3.1. FIRST USE
- 3.2. ADJUSTING THE INTENSITY
- 3.3. CHARGING
- 3.4. ATTACHMENTS

## 4. APPLICATION & USE

- 4.1. INTENDED USE, USER AND PATIENT
- 4.2. SELF-APPLICATION AT HOME
- 4.3. PROFESSIONAL APPLICATION IN NEUROREHABILITATION
- 4.4. PROF. APPLICATION IN VOICE AND SWALLOWING THERAPY

## 5. CONTRAINDICATIONS

## 6. TECHNICAL SPECIFICATIONS

- 6.1. SYMBOLS USED
- 6.2. TECHNICAL SPECIFICATIONS

## 7. IMPORTANT SAFETY INSTRUCTIONS

- 7.1. WARNINGS
- 7.2. OTHER PRECAUTIONS

## 8. MAINTENANCE AND REPAIR

- 8.1. CLEANING AND DISINFECTION
- 8.2. TROUBLESHOOTING
- 8.3. SAFETY CHECK, MAINTENANCE AND REPAIR
- 8.4. DISPOSAL

## 9. WARRANTY

## 10. CUSTOMER SERVICE

### 1. PREFACE

You have made an excellent choice with our vibration sound wave device - the DAS NOVAFON lite. Our therapy devices have delivered excellent performance and reliability for decades. Please read the following instructions carefully to ensure that you can get the most out of your NOVAFON vibration sound wave device and enjoy using it for a long time. The explanation of the footnotes can be found on the last page of this instructions.

## 2. PRODUCT DESCRIPTION

### 2.1. SCOPE OF DELIVERY

Delivery includes a DAS NOVAFON lite (see available variants chapter 2.2) including a disc attachment, a ball attachment, a power plug, a charging cable and instructions for use.

DAS NOVAFON lite also features a colour matched extension handle.

### 2.2. DEVICE TYPES

NOVAFON vibration sound wave devices are designed for local vibration therapy. Different attachments are used to transmit the mechanical vibrations into the tissue. Thanks to its safe and simple application, local vibration therapy can be used in many ways. Thanks to the three treatment frequencies (50 Hz, 75 Hz and 100 Hz) and three intensity settings (low, medium, high), treatment can be individually adapted to the body part and underlying symptoms in question.

The following table describes the characteristics of your new NOVAFON device, DAS NOVAFON Power 2.

The different device types differ in their color and the intensity of the vibrations (see table below):

DAS NOVAFON lite	
<b>Multi-function button</b>	ON/OFF, (🔊)
Frequency	100 Hz
Intensity/ strength setting	low (🔊), medium (🔊), high (🔊)
<b>Depth of penetration</b>	6 cm
<b>Colour</b> (only one colour included per delivery)	Forest [Art. Nr.: 1097-029] Bordeaux [Art. Nr.: 1097-030] Sand [Art. Nr.: 1097-031]
<b>Lithium-Ionen Akku</b>	Capacity 2400 mAh, voltage 3,7V, 8,8 Wh, Type ICR18650 with protection board

Article reference number and UDI/GTIN can be found on the bottom of the packaging box.

### 2.3. ACCESSORIES

NOVAFON vibration sound wave devices are equipped with two interchangeable attachments as standard. NOVAFON GmbH also offers

other attachments that are not included in this set. Please contact your dealer or visit [www.novafon.com](http://www.novafon.com) for more information. NOVAFON devices may only be used with original NOVAFON accessories.

THE USE OF ACCESSORIES FROM OTHER MANUFACTURERS CAN IMPAIR THE FUNCTION AND PERFORMANCE OF NOVAFON DEVICES.

#### 2.3.1. Disc attachment

The disc attachment is used in the treatment of larger body parts that require broader strokes. It is included in the standard delivery. Please note, that as a spare part the disc attachment might be available in a limited number of colours (e.g. white).

#### 2.3.2. Ball attachment

The ball attachment can be used for smaller areas and targeted treatment. It is included in the standard delivery. Please note, that as a spare part the ball attachment might be available in a limited number of colours (e.g. white).

#### 2.3.3. Extension handle

DAS NOVAFON lite is offered as standard in a package with a color-coordinated extension handle. The extension handle is attached to the device. When using for the first time, the extension handle's rocker must be pressed down to snap over the screw cover at the bottom of the device. The screw cover is pushed down slightly. This is a desired state.

To remove the extension handle again, press the indentation on the rocker down and pull the device and handle apart at the same time. Use both hands for this.

#### 2.3.4. Optional accessories

On our website you will find additional attachments for various treatment goals. There is also a charging stand and USB cover caps for your battery-powered NOVAFON device.

NOTE: Depending on your country the appliance might be additionally equipped with a second power plug with country specific connectors. These are available as spare parts, too.

#### 2.3.4. Combination with other products

NOVAFON devices are not intended to be used with medical products other than NOVAFON original accessories.

## 3. OPERATION

### 3.1. INITIAL START-UP

Fully charge the device prior to switching it on for the first time (chapter 3.3). Switch on the NOVAFON by pressing the ON/OFF button once. To switch off, press the button again.

### 3.2. ADJUSTING THE INTENSITY

Adjust the intensity by pressing the Intensity setting button repeatedly. You can select between three intensity levels: light (☺), medium (☺) and strong (☺). The intensity level is indicated by the LED display (intensity). The device always starts at the lowest intensity level (☺). This also corresponds to the NOVAFON treatment recommendation, i.e. starting with a low intensity and gradually increasing it according to need and personal comfort.

**NOTE:** The best vibration is achieved when the device is operated without additional contact pressure. If your device switches to a lower level by itself – or you're unable to increase the intensity – this is probably because your battery level is low. If this is the case, please recharge your device.

### 3.3. CHARGING

#### 3.3 CHARGING

To charge the device, only use the included power adapter and USB-C cable or the NOVAFON Charging Stand (not included). To charge the device, first connect the cable to the device or to the charging stand and then to the power adapter suitable for your power connection. Only then plug the mains adapter into the socket. Charge the device until the charge display (LED) no longer flashes. Do not use the NOVAFON device during charging. As soon as the LED display no longer flashes, the NOVAFON device is fully ready to use. The device has a low battery display. If the charge display (LED) starts flashing rapidly, you can continue using your NOVAFON device for another 10 minutes. The device then switches off automatically. If you use the charging stand for charging, make sure to place the appliance carefully on the USB port. When removing, hold the charging stand down and pull the device out carefully. The device is charging when the display (LED) lights up. Avoid twisting the appliance in the charging stand as this will prevent damage. Ensure that devices are completely dry and with no cleaning agent, disinfectant or oil/ cream residues before using the charging stand or charging cable.

### 3.4. ATTACHMENTS

The attachments are fixed to the device via a plug connection. They can easily be detached by turning them a quarter turn and pulling them gently off the device. The attachments click simply onto the device. The attachments are in the correct position when the brackets on the device and the attachment lie seamlessly on top of each other. The magnets help you to find the final position. Before starting treatment, make sure that the respective attachment is engaged in the intended final position.

Please refer to the drawing on page 21 for the correct position of the attachment on the device.

### 4. APPLICATION & USE

The local vibration therapy with the NOVAFON vibration sound wave device can be used as an aid in treating a number of diseases of the musculoskeletal and the nervous system. NOVAFON vibration sound wave devices are used by doctors and therapists. Due to their simple handling, the devices can also be used in the home environment. Please bear in mind that NOVAFON vibration sound wave devices can only alleviate symptoms but never actually cure the underlying disease, nor are they a replacement for a doctor's visit. Despite being safe, simple and non-invasive, we recommend that you discuss the treatment with your doctor or therapist. If symptoms get worse, seek medical advice. Discontinue the treatment immediately, if you experience nausea, dizziness, increased pain, or skin irritation. Please be aware of the contraindications listed in section 5.

#### 4.1 INTENDED USE, USER AND PATIENT

DAS NOVAFON lite is designed mainly for private use at home.

NOVAFON devices and accessories are designed to relieve pains and aid in relaxation of muscles to alleviate symptoms of:

- Temporary and transient pain of the musculoskeletal system<sup>1</sup>
- Chronic pain of the musculoskeletal system<sup>1</sup>
- Neuro-rehabilitation<sup>2,3</sup>
- Voice and swallowing therapy<sup>4</sup>

Please note, that some use scenarios might require the use of special NOVAFON accessories (separately available).

DAS NOVAFON lite is particularly suited for pain treatment.

Devices and accessories can be used head to toe. **NOT TO BE USED ON GENITALS.**

Devices can be used on patients from the age of 6.5 When supervised by professionals, earlier use can be feasible. However, **DO NOT USE** on patients younger than 2 years of age.

#### 4.2. SELF-APPLICATION AT HOME

Taking into account the contraindications (chapter 5), move the NOVAFON device in a straight line, selectively or in slowly circling movements on or around the treatment area. The NOVAFON device can also be applied at trigger points. In case of uncertainty, please consult your treating physician or therapist regarding your individual application. The NOVAFON device can be used directly on the skin or through fabric / clothing. During the application, make sure that the vibration sound wave device glides gently and without pressure over the body area. Use the entire surface of the attachment. This is the only way the penetration depth effect of the NOVAFON device can be fully deployed and the vibrations are transmitted optimally. Use only the NOVAFON device's own weight for a comfortable application.

The therapeutic application shall not exceed a maximum of three 20-minute treatment units per day. Please note that the NOVAFON device must be switched off for 15 minutes after 20 minutes of use

in order to avoid overheating. When treating very painful or sensitive parts of the body (e.g. face, neck, head), the duration of application should be reduced and adjusted to suit the patient's personal needs. TIP: You can save your favourite frequency and intensity settings in the app and call them up directly each time you use your device.

Treatment should not exceed a maximum of 20 minutes per session, nor a maximum of 3 sessions per day. Please note that after 20 minutes of use, the NOVAFON should be switched off for 15 minutes in order to avoid overheating. The device automatically switches to stand-by mode after 20 minutes of continuous use. When treating severely painful or sensitive parts of the body (e.g. face, neck, head), you should reduce / adjust the duration of treatment according to each individual's sensitivity.

NOTE: Application of liniments: Apply only a thin layer of creams, ointments or cosmetic / medicinal liniments to the skin, or place a fine cloth between the device and the skin. Do not allow liquids or residues to penetrate the NOVAFON as this may cause damage.

After using agents of this type, make sure you clean the device thoroughly.

NOTE: Discontinue treatment immediately if side effects occur (chapter 5).

#### 4.3. PROFESSIONAL USE IN NEUROREHABILITATION

NOTE: The use of local vibration therapy for neurological diseases or voice and swallowing therapy should only be carried out by specialists (therapists, physicians, nurses) or after detailed instruction by such a person.

##### 4.3.1. Spasticity

Spasticity is a common consequence of a stroke and other disorders of the central and peripheral nervous system. Local vibration therapy, combined with physiotherapy, provides a simple and drug-free treatment method. Numerous studies have shown a reduction in spasticity as well as improved motor control and function.<sup>2</sup> The effectiveness mechanisms are the tonic vibrational reflex and the stimulation of the sensorimotor system. Taking into account the contraindications (section 5), the spastic muscle itself is treated at 50 Hz and the antagonist of the spastic muscle at 100 Hz. The starting point can be on the tendon or directly on the muscle itself. The treatment may be local, i.e. focusing on one point, or in sweeping movements. The application with the NOVAFON device shall not exceed 20 minutes per unit. The treatment can be performed up to 5 times a week. Please keep in mind that using the appliance incorrectly can lead to increased muscle tone. Stop the treatment immediately if any side effects occur (see section 5).

##### 4.3.2. Hemineglect

Several studies have found an improvement in the visual exploration

performance of stroke patients with Hemineglect when the neck musculature contralateral to the lesion is stimulated by vibration.<sup>3</sup> This effect is based on the fact that the vibration stimulates stretch receptors in muscles and tendons of the neck, leading to an apparent rotation of the head. For successful therapy, the corresponding stimulation point must be found on the contralesional, posterior neck muscles, taking into account the contraindications (section 5). This point is determined by fixing a bright spot on the wall (e.g. by means of a laser pointer) in a darkened room. The therapist guides the NOVAFON device slowly along the neck muscles at 100 Hz and medium intensity. The patient sits on a chair for the treatment. The correct stimulation point is determined as soon as the patient reports that the fixed point is moving without moving his / her head. The total treatment time should not exceed 40 minutes, whereby the pure stimulation time should be interrupted by a 5 minute break after 10 minutes. A total of at least 15 treatment sessions are necessary to achieve a lasting effect. Stop the treatment immediately if any side effects occur (see section 5).

#### 4.4. PROFESSIONAL APPLICATION IN VOICE AND SWALLOWING THERAPY

NOTE: The use of local vibration therapy for swallowing and voice disorders must only be carried out by specialists (therapists,

#### 5. CONTRAINDICATIONS & SIDE EFFECTS

Treatments with the NOVAFON vibration sound wave devices are not permitted for patients with:

- Open wounds / eczema (in the body region to be treated)
  - Arteriosclerosis (in the body region being treated)
  - Current episodes of epilepsy
  - Pregnancy
  - Implants (in the body region being treated)
  - Heart pacemakers
  - Thromboses (in the body region being treated)
  - Brain pacemakers
  - Cardiac arrhythmia
  - Tumours, cancerous and not cancerous
  - Acute inflammations (in the body region being treated)
  - Acute episodes of inflammatory diseases
- Side effects:

Please note that in rare cases the following side effects may occur:

- Increased pain / unpleasant sensation
- Nausea / dizziness
- Skin irritation
- Reflex / involuntary muscle contractions

Please discontinue the treatment immediately, if you experience any of these side effects. If symptoms persist, please consult a doctor.

NOTE: Pain treatment in particular can lead to an initial worsening of the symptoms before you notice any improvement. NOTE: Slight reddening and warming is not a cause for concern. This is a desired effect of local vibration therapy, which is caused by the increased blood flow.

DO NOT USE for treating pains or swellings of unknown origin. In that case consult a medical professional for clarification and diagnosis.

## 6. TECHNICAL SPECIFICATIONS

### 6.1. SYMBOLS USED

~ Alternating current (AC)

≡ Direct current (DC)

 Device with protection class II according to DIN EN 61140

Hz Hertz

KB Short-term operation [SO]

mA Milliampere-hour

A Amp

V Volt

 Disposal according to the EU Directive on Waste Electrical and Electronic Equipment (WEEE)

 Displays the manufacturer of the medical device

 Displays the serial number as defined by the manufacturer to identify a specific device.

 Protection class of the application part: type BF

 Temperature limit: Refers to the temperature limits to which the medical device can be safely exposed.

 Humidity limit

 Atmospheric pressure limit

 Refer to the instructions

 Use inside only

 General warning symbols

 Intensity setting in 3 stages

### 6.2. TECHNICAL SPECIFICATIONS

· Operating conditions: 5 °C [41 °F] to 30 °C [86 °F]; 15 % to 93 % relative humidity, non-condensing; 700 hPa to 1060 hPa air pressure

· Storage / transport: -5 °C [23 °F] to 40 °C [104 °F]; 15 % to 93 % relative humidity, non-condensing; 500 hPa to 1060 hPa air pressure

· Operating mode: SO 20/15 min

· Operating noise: maximum 70 dBA

· Power plug charger 1180004 (white)

· Input power: AC 100–240 V, 50–60 Hz, 300 mA

· Output power: DC 5 V, 2 A

· The standard scope of delivery includes a type-F power plug [EU plug]. Additional power adapters with other connection

plugs are available as options or are already included in the relevant country sets.

· Charging cable USB-A to USB-C, length: approx. 1 m [3ft]

· Battery: Lithium-ion battery, service life: 800 charging cycles. With a new battery and average intensity, one battery charge guarantees at least six treatment units of 20 minutes each. Note that frequent use of the highest intensity level may reduce that time span.

· Battery performance data: Capacity 2400 mAh. Voltage 3.7 V, type ICR18650 with protective PCB

· The devices comply with IEC 60601-1-2:2014 or CISPR 11 emission class B and group 1. The devices have been tested with the interference immunity test levels for home use.

· Product category: Class IIa medical device according to the Medical Device Regulation (2017/745/EU), furthermore conformity with 2015/863/EU is declared. FDA class I product Codes ISA and IRO.

· IP22 protection class, which states: Protected against access with a finger. Protected against water dripping at an angle (when using the USB cover caps, not included in standard scope of delivery).

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## 7. IMPORTANT SAFETY INSTRUCTIONS

Before starting to use the device, please read the user manual and these safety instructions carefully and strictly follow them.

### 7.1. WARNINGS

When using an electrical device basic precautions should always be followed including the following to reduce the risk of burns, fires, electric shock, possibly severe injury and possible death to the user or other people:

WARNING: Do not use the NOVAFON device during charging.

WARNING: Always store the device in a dry and clean location.

WARNING: Before connecting the power plug to the mains power source, check that the mains voltage corresponds to the voltage specification on the power plug. Connect the NOVAFON device only to a power outlet that delivers the correct voltage.

WARNING: To charge the device, use only the charging cable and power supply included in the delivery or the separately available charging stand. Use the correct connector plug, if applicable.

WARNING: Always unplug the device and all charging accessories from the electrical outlet after charging and before cleaning.

WARNING: Ensure that devices are completely dry and with no cleaning agent, disinfectant or oil / cream residues before using the charging stand or charging cable.

WARNING: Do not bring the device and charging accessories (plug, charging cable, charging stand) into contact with hot surfaces (e.g. heater, hotplate, etc.). The housing of the device or adapter or the

insulation of the cable could be damaged.

WARNING: Do not use the device when taking a shower or bath.

EN 28

WARNING: Do not operate where aerosol (spray) products are being used, in other explosive atmospheres or where oxygen is being administered.

WARNING: Do not place or drop the device into water or other liquids.

WARNING: Do not reach for a device that has fallen into water, especially when the charging cable is connected to a power outlet. Unplug immediately.

WARNING: Do not place or store device where it can fall or be pulled into a tub, pool or sink.

WARNING: Protect the device, charger and charging stand from drop impact and shocks.

WARNING: This device may not be modified without the manufacturer's permission.

WARNING: NOVAFON devices may only be used with original NOVAFON accessories. The use of accessories from other manufacturers can impair the function and performance of NOVAFON devices. This also applies to the power adapter and charging cables.

WARNING: Do not use the device and either discard or return to your dealer for examination if any of the following conditions occur:

- if the device fails to work correctly.
- if the device is damaged, e.g. after having been dropped
- if the device was immersed in water or other liquids
- if the device exhibits other safety-related problems, such as emitting smoke, unusual or excessive heat etc.

WARNING: Do not use the power plug, charging cable or charging stand:

- in the event of visible damage or after dropping the power plug, charging cable or charging stand
- if the power plug, charging cable or charging stand has been exposed to extreme moisture
- if liquid has penetrated the unit

WARNING: Never charge the device for more than 24 hours at a time. Do not leave the device unattended. Disconnect the device from the power supply at night. This also applies when using the charging stand.

WARNING: Do not place the device on flammable surfaces when charging.

WARNING: Never leave the device unattended when it is turned on.

WARNING: Do not clamp or pinch any part of your body, especially your fingers, between the attachment and the device. During the treatment, hold the device only by the device handle.

WARNING: Never drop or insert any foreign object into any opening on this device. Do not insert any objects (including wires) into openings in the device.

WARNING: Wireless communication devices (such as Wi-Fi stations, mobile phones, cordless phones, and related base stations) can affect the device and should be kept at least 0.5 m (2 feet) away from the device during use. This also applies to cables and plug-in power supplies.

WARNING: Careful supervision is necessary when used on or by children, immobile or disabled individuals. NOVAFON devices should only be used by users from the age of 18 years who have read the instructions and are competent to use the device safely.

WARNING: Do not use on sleeping or unconscious individuals.

WARNING: Keep the device out of reach of children when not in use.

WARNING: Small parts (e.g. cover caps) could come loose and be swallowed. Risk of suffocation!

WARNING: Keep children away from packaging materials. There is a risk of suffocation.

WARNING: Do not use on genitals, this device is not intended as an intimate pleasure toy.

WARNING: Do not use outdoors or in wet or damp areas.

WARNING: Never operate the power plug or charging stand in environments with high humidity or in the presence of moisture. Please note, that moisture is, for example, also generated by nebulisers.

WARNING: Do not operate under blankets, pillows, towels, clothing etc. or anywhere where ventilation is limited as the device could over-heat and thus cause fire, electric shock or other injuries to people.

WARNING: Use the device only for its intended use as described in this manual.

WARNING: Avoid contact with mucous surfaces or the eyes.

WARNING: Do not use the device together with other medical devices, as this may affect the performance and function of the device as well as the other product.

## 7.2 OTHER PRECAUTIONS

NOTE: Use the initial session to figure how the body responds to vibration therapy. If you experience any discomfort, dizziness or nausea stop session immediately.

NOTE: Do not use the device if any of the contraindications described in chapter 5 apply to you.

NOTE: Do not use the device for more than 20 minutes per application. Leave it to cool down for at least 15 minutes before restarting.

NOTE: Make sure to secure the attachments fully onto the NOVAFON device and keep it on during treatment.

NOTE: Make sure that no substances, such as liquids, creams or the like, can penetrate into the device or the power plug.

NOTE: To prevent damage, make sure that the device and charging stand are standing or resting on a stable surface during the charging process.

NOTE: Do not twist the device in the charging stand. This may cause damage.

NOTE: Protect the device against dust, lint and other particles. Penetration of dust, lint or other particles may cause the device to fail.  
NOTE: To prevent premature wear, only store the device in a dry and clean place that is protected against UV light (incl. sunlight).

NOTE: Use only non-alcoholic cleaning supplies. Alcohol can attack the surfaces of the housing and the attachments.

EN 30

## 8. MAINTENANCE AND REPAIR

### 8.1. CLEANING AND DISINFECTION

Do not clean the device during charging. Disconnect the device from the power supply when cleaning. If used privately, clean the housing and attachments using a clean cloth moistened with fresh water or a mild soapy solution. For clinical and professional use, clean and disinfect device housings and all attachments with an appropriate surface disinfectant suitable for medical devices before and after each use. As a general rule, only use alcohol-free disinfectants that are suitable for use on sensitive surfaces, such as NOVACLEANER by NOVAFON, SinAlc sensitive by HIMED or RHEOSEPT-WD plus wipes by Rheosol.

If necessary, dry off with a soft cloth.

If you are using the charging stand, this can be cleaned with a clean cloth moistened with fresh water or a mild soapy solution if necessary. Do not clean the USB port. For professional use: To prevent contamination, only place cleaned devices in the charging stand. Ensure that devices are completely dry and with no cleaning agent, disinfectant or oil / cream residues before using the charging stand or charging cable.

### 8.2. TROUBLESHOOTING

If your device automatically switches to a lower level or you are unable to increase the intensity, this is probably due to a low battery level. In such a case, please charge the device.

You can also use the "FAQ" area (possibly download document) of our website to find helpful answers to frequently asked questions.

If there are any problems with the device, please contact our customer service. Never open the device yourself to repair malfunctions. If you attempt to repair the device yourself, you will void the warranty. The manufacturer declares conformity with the requirements in respect to medical devices of class IIa according to Medical Device Regulation 2017/745/EU in sole responsibility for this product. Please contact us in case you suspect the device to have caused a serious incident. As a user you have the right to report such an incident to your national competent authority (e.g. Ministry of health; FDA). You can also use the "FAQ" section (download document if necessary) of our website to find helpful answers to frequently asked questions.

### 8.3. SAFETY CHECK, MAINTENANCE AND REPAIR

The NOVAFON device does not require regular inspection or maintenance. To ensure safe operation, however, you should check the

device regularly for obvious damage. We also recommend that you carry out an electrical safety test, e.g. compliant with DGUV 3 (German Social Accident Insurance), at least every 2 years. Please note that there may be national laws that oblige you to carry out safety checks on the device. This does not generally apply to private individuals. If you have any problems with your appliance either discard or return it to your dealer. Attachments shall be replaced once they show clear signs of wear or damage (cracks, deformations, change in surface). Repairs can only be carried out by NOVAFON itself or by qualified dealers. Information regarding repair options and spare parts can be found on our website.

NOTE: Attachments are wear parts and are therefore facing a limited warranty.

### 8.4. DISPOSAL

Devices must never be disposed of with common household waste. For environmentally sound disposal, the user is obligated to dispose of the old device at a designated collection point for the disposal of electrical and electronic equipment. Please observe the national legal regulations when disposing of the device. You can also send your old device to NOVAFON; we will gladly dispose of it for you. This device is subject to EU Directive on Waste Electrical and Electronic Equipment (WEEE).

To ensure the conservation of resources, please dispose of the packaging separately as waste paper

## 9. WARRANTY

From the date of delivery, we grant a warranty of 2 years (24 months) for defects resulting from material or production defects. Warranty coverage is subject to the presentation of a proof of purchase. The battery, charging cable, power plug and the attachments is guaranteed for 6 months.

Excluded from the warranty: Damage due to improper use, normal wear and tear and consumption (also valid for accessories).

The right of rescission (reversal of the purchase), reduction (discount on the purchase price) or subsequent delivery is initially excluded. However, you have a claim to reduction or conversion at your discretion if we do not remedy a defect that is subject to our warranty within a reasonable period of time. Any damage caused by failure to use the product in accordance with its intended use or resulting from improper or unreasonable use or maintenance is not covered by the warranty. Such damage is the sole responsibility of the user. Failure to comply with this instruction manual will void the warranty. Some states may not allow such limitation, so it may not apply to you. This limited warranty gives you specific and you may have other rights which may vary from state to state. Get in touch with your dealer or supplier to claim warranty.

## 10. CUSTOMER SERVICE

The device may only be repaired by NOVAFON Customer Service. Please do not send in your defective device without being asked, but

rather clarify the damage and repair costs with us first. Repair could be more expensive than a new device. If the repair is no longer financially feasible within the warranty period, NOVAFON offers a replacement unit (again with a 24-month warranty).

This manual can also be viewed as a PDF file at [www.novafon.com](http://www.novafon.com). Failure to comply with this instruction manual will void the warranty. Some states may not allow such limitation, so it may not apply to you. This limited warranty gives you specific and you may have other rights which may vary from state to state.

Get in touch with your dealer or supplier to claim warranty.

## 10. CUSTOMER SERVICE

The device may only be repaired by NOVAFON Customer Service. Please do not send in your defective device without being asked, but rather clarify the damage and repair costs with us first. Repair could be more expensive than a new device. If the repair is no longer financially feasible within the warranty period, NOVAFON offers a replacement unit (again with a 24-month warranty).

This manual can also be viewed as a PDF file at [www.novafon.com](http://www.novafon.com).

## Notes

- <sup>1</sup> **Serritella et al.** Local Vibratory Stimulation for Temporomandibular Disorder Myofascial Pain Treatment: A Randomised, Double-Blind, Placebo-Controlled Preliminary Study. *Pain Research Management. Pain Res Manag.* 2020 Dec 5;2020:6705307.
- Staud et al.** Attenuation of Experimental Pain by Vibro-Tactile Stimulation in Patients with Chronic Local or Widespread Musculoskeletal Pain.
- Beinert et al.** After-effects of neck muscle vibration on sensorimotor function and pain in neck pain patients and healthy controls - a case-control study. *Disabil Rehabil.* 2019 Aug;41(16):1906-1913.
- Guieu et al.** Substance P-like immunoreactivity and analgesic effects of vibratory stimulation on patients suffering from chronic pain. *Can J Neurol Sci.* 1993 May;20(2):138-41.
- Lundeberg T.** Vibratory Stimulation for the Alleviation of Pain. *The American Journal of Chinese Medicine*, 1984, Summer, 23 (1-4): 60-70.
- Lundeberg T.** Long-term results of vibratory stimulation as a pain relieving measure for chronic pain. *Pain.* 1984 Sep;20(1):13-23. doi: 10.1016/0304-3959(84)90807-8
- Lundeberg T.** The pain suppressive effect of vibratory stimulation and transcutaneous electrical nerve stimulation (TENS) as compared to aspirin. *Brain Res.* 1984 Mar 5;294(2):201-9.
- Lundeberg T.** Relief of pain from a phantom limb by peripheral stimulation. 1985. *Journal of Neurology*; 232(2):79-82.
- <sup>2</sup> **Wanderley et al.** Rebelatto JR. Effect of plantar vibration stimuli on the balance of older women: a randomized controlled trial. *Arch Phys Med Rehabil.* 2011 Feb;92(2):199-206. doi: 10.1016/j.apmr.2010.10.014. PMID: 21272715.
- NOVAFON GmbH.** Results use observation occupational therapy stroke 2020 (unpublished).
- Alashram et al.** Effectiveness of focal muscle vibration on hemiplegic upper extremity spasticity in individuals with stroke: A systematic review. *NeuroRehabilitation.* 2019 Dec 18;45(4):471-481.
- Celletti C, Camerota F.** Preliminary evidence of focal muscle vibration effects on spasticity due to cerebral palsy in a small sample of Italian children. *Clin Ter.* 2011;162(5):e125-8.
- Mandic et al.** Kinematic analysis of upper-extremity movements after segmental muscle vibration therapy. *Gait & Posture.* 35: S21 - S22. *Gait & Posture*, 2012; 35: 21-22
- Mortaza et al.** Upper limb tendon/ muscle vibration in persons with subacute and chronic stroke: a systematic review and meta-analysis. *Eur J Phys Rehabil Med.* 2019 Oct;55(5):558-569.
- <sup>3</sup> **Johannsen et al.** Lasting amelioration of spatial neglect by treatment with neck muscle vibration even without concurrent training. *J Rehabil*

Med. 2003 Nov;35(6):249-53.

**Kamada et al.** Effects of 5 minutes of neck-muscle vibration immediately before occupational therapy on unilateral spatial neglect. *Disabil Rehabil.* 2011;33(23-24):2322-8.

**Karnath et al.** The perception of body orientation after neck-proprioceptive stimulation. Effects of time and of visual cueing. *Exp Brain Res.* 2002 Apr;143(3):350-8

<sup>4</sup> **Barsties V, Latoszek B.** Preliminary study of Novafon local vibration voice therapy for dysphonia treatment. *Logoped Phoniatr Vocol.* 2020 Apr;45(1):1-9.

**Barsties v. Latoszek.** Treatment Effectiveness of Novafon Local Vibration Voice Therapy for Dysphonia Treatment. *Journal of Voice,* Volume 34, Issue 1, 2020, Pages 160.e7-160.e14

**Barsties v. Latoszek B, Watts CR.** A Case of Nervus Laryngeus Superior Paresis Treated With Novafon Local Vibration Voice Therapy, *Journal of Voice,* Volume 35, Issue 3, 2021, Pages 406-410.

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**Grant L.** The use of a manual vibrator in the speech therapy program of four school-age mentally retarded children. *J Commun Disord.* 1982 Sep;15(5):375-83.

<sup>5</sup> **Eklund G, Steen M.** Muscle vibration therapy in children with cerebral palsy. *Scand J Rehabil Med.* 1969;1(1):35-7.

**Celletti C, Camerota F.** Preliminary evidence of focal muscle vibration effects on spasticity due to cerebral palsy in a small sample of Italian children. *Clin Ter.* 2011;162(5):e125-8

**Grant L.** The use of a manual vibrator in the speech therapy program of four school-age mentally retarded children. *J Commun Disord.* 1982 Sep;15(5):375-83.

More Literature can be found on our Website.

Other languages of this manual are available online.