novafon®

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English

1. Preface	10
2. Scope of delivery	10
3. Applications & use	10
3.1. Intended use	11
3.2. Typical applications	11
4. Important instructions	12
5. Material information	13
6. Product Life	13
7. Cleaning and disinfection	14
8. Contraindications / side effects	14
9. CE Conformity	15
10. Contact	16

1. Preface

The attachments of the novafon ‘Reha Set one’ are designed as an extension to the existing attachments supplied with the novafon sound wave devices for local vibration therapy. They are intended solely for use in conjunction with such a device. Please read the following pages carefully to ensure proper use.

2. Scope of delivery

The scope of delivery includes a cone attachment one, a pin attachment one, a disc attachment XL one and instructions for use. The Rehabilitation Set one is available in white or black (item no. 7016-040 / 7016-007)

3. Applications & use

The attachments are fixed to the device via a plug connection. The attachments can easily be detached by turning them a quarter turn and pulling them gently off the device. The attachments click simply onto the device. The attachments have in the correct position when the brackets on the device and the attachment lie seamlessly on top of each other. The magnets help you to find the final position. Before starting treatment, make sure that the respective attachment is engaged in the intended final position.

Note: The attachments can be used directly on the skin or through fabric/clothing. There is no need for gel or similar.

3.1. Intended use

- novafon devices and accessories are designed to relieve pains and aid in relaxation of muscles to alleviate symptoms of:
- Temporary and transient pain of the musculoskeletal system
 - Chronical pain of the musculoskeletal system
 - Neuro Rehabilitation (sensorimotor disorders)
 - Voice and swallowing disorders

3.2. Typical applications

Cone attachment one	Pin attachment one	Disk attachment XL one
Targeted treatment for trigger and pain points	Slow movements for additional sensory input	Large area treatment, e.g. , back and thighs

Cone attachment one	Pin attachment one	Disk attachment XL one
Precise targeting of tissue such as scar tissue	Rotational movements to promote circulation	Sensory training from head to toe
Treatment of fine structures on the face, hands and feet	Treatment of pain on the head/hairline via brush-like surface	Full body treatment after exercise

Note: Vibration therapy primarily relieves symptoms, but does not cure the underlying disease. The use of the Rehabilitation Set does not replace a visit to the doctor.

Note: The use of local vibration therapy for muscle relaxation in neurological diseases or voice and swallowing therapy should only be carried out by specialists (therapists, physicians, nurses) or after detailed instruction by such a person.

In all cases Local Vibration Therapy is used as one element in a wholistic therapy plan where muscle relaxation is needed for further exercises and training elements. novafon devices can be used on people from the age of 6. Children need to be supervised. The same may apply to people with limited cognition or motor control (as often found in patients with spasticity). MIND THE CONTRAINDICATIONS as named in chapter 8 as well as the instructions in chapter 4.

DO NOT USE for treating pains or swellings of unknown origin. In that case consult a medical professional for clarification and diagnosis.

4. Important instructions

- Depending on personal sensitivity, the vibration can be perceived with different intensity. Treat particularly sensitive spots at the lowest intensity level, through clothing or a fine cloth.
- Treat a maximum of 20 minutes per unit, then let the novafon sound wave device cool down for 15 minutes.
- In case of discomfort, dizziness, or an increase in the symptoms, stop the treatment.
- Keep attachments out of reach of children.
- Do not use the attachments on mucous membranes and open wounds.
- Avoid impacts! These can damage the products and hence lead to a risk of injury.
- novafon accessories are only intended for use with novafon vibration sound wave devices. Use with products from other manufacturers is not permitted.
- When using these attachments, the noise generated may differ from that of the standard attachments. Do not switch on the device until you have made physical contact.
- Before using the attachments, please also observe all information from the operating instructions of your novafon sound wave device.

5. Material information

The novafon rehabilitation attachments are made of high-quality plastic suitable for medical applications. Please store in a cool, dry place away from sunlight.

6. Product Life

The product life of the attachments will vary depending on the frequency of use. Check the attachments before each application. Replace damaged or deformed products. We recommend replacing the attachments after two years of use. Even much sooner if used frequently. Please note that attachments are considered wear parts and are therefore subject to a limited warranty.

7. Cleaning and disinfection

For professional and clinical use, novafon attachments must be disinfected with a surface disinfectant for medical devices before and after each application. For cleaning, please use alcohol-free disinfectants that are suitable for use on sensitive areas, such as novacleaner from novafon, SinAlc sensitiv from Himed or Rheosept-WD plus Wipes from Rheosol. If necessary, wipe it dry with a soft cloth. In case of dirt or if otherwise necessary, clean the attachments with a brush and a neutral cleaner under running water before disinfecting. Especially for the pin-attachment, we recommend an alcohol-free disinfectant spray. For domestic, non-clinical use, clean the attachments with a damp cloth, under running water or in a mild soapy solution. Allow the attachments to dry completely before using them with the appliance again.

8. Contraindications/side effects

Treatments with the novafon sound wave devices are not permitted for patients with:

- Open wounds/eczema (in the body region to be treated)

- Arteriosclerosis (in the body region being treated)
- Current episodes of epilepsy
- Pregnancy
- Implants (in the body region being treated)
- Heart pacemakers
- Thromboses (in the body region being treated)
- Brain pacemakers
- Cardiac arrhythmia
- Tumours, cancerous and not cancerous
- Acute inflammations (in the body region being treated)
- Acute episodes of inflammatory diseases

Side effects:

Please note that the following side effects may occur:

- Increased pain/unpleasant sensation (occasionally)
- Discomfort/dizziness/restlessness (occasionally)
- Skin irritation (rarely)
- Bruises (very rarely)
- Reflexes/involuntary muscle contractions (rarely)
- Muscle soreness (rarely)

Please discontinue the treatment immediately, if you experience any of these side effects. If symptoms persist, please consult a doctor. Please note that pain treatment in particular can lead to an initial worsening of the symptoms before you notice any improvement. Please note that slight reddening and warming is not a cause for concern. This is a desired effect of local vibration therapy, which is caused by the increased blood flow.

9. CE Conformity

The manufacturer declares conformity with the requirements in respect to medical devices of class I according to Regulation 2017/745/EU and UK MDR 2002 in sole responsibility for this product. Please contact us in case you suspect the device to have caused a serious incident. As a user you have the right to report such an incident to your national competent authority (e.g Ministry of health; FDA).

10. Contact

If you have any questions, please contact your local dealer or the manufacturer.

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